

FOR INSTITUTIONAL AND COMMERCIAL USE ONLY

EPA Est No. 65787-MD-001

EPA Reg No. 65787-2

Net Contents: 1.1 oz.

Bleach Disinfectant Wipe

CAT # 09041

1:10 DILUTION

5,000 ppm Available Chlorine

Active ingredient:

Sodium hypochlorite: 0.525%

Other ingredients: 99.475%

Total: 100.000%



ALCAVIS
BLEACH-WIPE

For wiping hard non-porous environmental surfaces and patient care equipment

KEEP OUT OF REACH OF CHILDREN

CAUTION

FIRST AID

If in eyes:	<ul style="list-style-type: none">• Hold eye open and rinse slowly and gently with water for 15 – 20 minutes.• Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
If on skin or clothing:	<ul style="list-style-type: none">• Take off contaminated clothing.• Rinse skin immediately with plenty of water for 15 – 20 minutes.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. In an emergency, call a doctor or Poison Control Center at 1-800-222-1222

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION Harmful if absorbed through skin. Liquid may cause moderate eye irritation. Avoid contact with skin, eyes or clothing. Wear gloves for sensitive skin or prolonged use since it may cause skin irritation. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

Tuberculocidal and Bactericidal
1 minute minimum contact time
Maximum surface area that can be
disinfected per towelette is 3' x 3'

ALCAVIS BLEACH-WIPE

PREMOISTENED 8" x 10" TOWELETTE



LICENSED

PERIOD 2015-2017 LIC. NO.

7015.1

Suggested areas of usage:

Medical, Dental and Laboratory Counters, Exam Tables, Carts, Point of Care Equipment, Telephones, Sink Tops.

Directions for Use:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

- Always use personal protective equipment (gloves).
- Open wipe packet.
- Remove premoistened towelette.
- Apply towelette to hard non-porous surface to be disinfected.
- Thoroughly wet surface making sure treated surface remains visibly wet for 1 full minute.
- Use additional wipe if needed to assure continuous 1 minute wet contact time.
- Allow surface area to air dry and discard used towelette.
- Do not re-use used towelette.

Do not use this product as a terminal sterilant/high level disinfectant on any surface or instrument that is introduced directly into human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

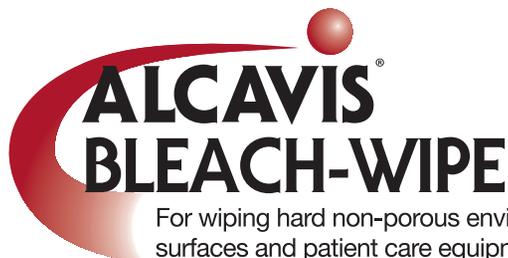
Bleach-Wipe is a 1:10 dilution of 5.25% sodium hypochlorite solution with filtered water to comply with recommended infection control protocol.

STORAGE AND DISPOSAL:

Do not contaminate water, food, or feed by storage or disposal.

Storage: Store this product in a cool, dry and well-ventilated area away from direct sunlight and heat. **Towelette Disposal:** Do not reuse towelette.

Dispose of used towelette and empty pack in trash. Do not flush in toilet.

**Manufactured by:**

Angelini Pharma Inc.
8322 Helgerman Court
Gaithersburg, MD 20877
1 (800) 726-2308

BOX LABEL



BLEACH WIPES – SINGLES

Catalog # 09041

ANGELINI 5,000 PPM Available Chlorine
TUBERCULOCIDAL & BACTERICIDAL

Lot #

1 wipe per Packet; 100 Packets per Box

Net Contents of Box: 6 lbs., 14 oz.



FOR INSTITUTIONAL AND COMMERCIAL USE ONLY
EPA Est No. 65787-MD-001 EPA Reg No. 65787-2 Net Contents: 1.1 oz.

Bleach Disinfectant Wipe **CAT # 09041**
1:10 DILUTION
5,000 ppm Available Chlorine

ALCAVIS BLEACH-WIPE
For wiping hard non-porous environmental surfaces and patient care equipment

Active ingredient: Sodium hypochlorite: 0.525%
Other ingredients: 99.475%
Total: 100.000%

**KEEP OUT OF REACH OF CHILDREN
CAUTION**

FIRST AID	
If in eyes:	<ul style="list-style-type: none"> Hold eye open and rinse slowly and gently with water for 15 – 20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
If on skin or clothing:	<ul style="list-style-type: none"> Take off contaminated clothing. Rinse skin immediately with plenty of water for 15 – 20 minutes.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. In an emergency, call a doctor or Poison Control Center at 1-800-222-1222

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals
CAUTION Harmful if absorbed through skin. Liquid may cause moderate eye irritation. Avoid contact with skin, eyes or clothing. Wear gloves for sensitive skin or prolonged use since it may cause skin irritation. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

ALCAVIS BLEACH-WIPE
PREMOISTENED 8" x 10" TOWELETTE

Tuberculocidal and Bactericidal
1 minute minimum contact time
Maximum surface area that can be disinfected per towelette is 3' x 3'

Suggested areas of usage:
Medical, Dental and Laboratory Counters, Exam Tables, Carts, Point of Care Equipment, Telephones, Sink Tops.

- Directions for Use:**
It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.
- Always use personal protective equipment (gloves).
 - Open wipe packet.
 - Remove premoistened towelette.
 - Apply towelette to hard non-porous surface to be disinfected.
 - Thoroughly wet surface making sure treated surface remains visibly wet for 1 full minute.
 - Use additional wipe if needed to assure continuous 1 minute wet contact time.
 - Allow surface area to air dry and discard used towelette.
 - Do not re-use used towelette.

Do not use this product as a terminal sterilant/high level disinfectant on any surface or instrument that is introduced directly into human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Bleach-Wipe is a 1:10 dilution of 5.25% sodium hypochlorite solution with filtered water to comply with recommended infection control protocol.

STORAGE AND DISPOSAL:
Do not contaminate water, food, or feed by storage or disposal.
Storage: Store this product in a cool, dry and well-ventilated area away from direct sunlight and heat. **Towelette Disposal:** Do not reuse towelette. Dispose of used towelette and empty pack in trash. Do not flush in toilet.

ALCAVIS BLEACH-WIPE
For wiping hard non-porous environmental surfaces and patient care equipment



Manufactured by:
Angelini Pharma Inc.
8322 Helgerman Court
Gaithersburg, MD 20877
1 (800) 726-2308

CASE LABEL

BLEACH WIPES – SINGLES

Catalog # 09041

5,000 PPM AVAILABLE CHLORINE
TUBERCULOCIDAL & BACTERICIDAL

Lot #



FOR INSTITUTIONAL AND COMMERCIAL USE ONLY
EPA Est No. 65787-MD-001 EPA Reg No. 65787-2 Net Contents: 1.1 oz.



CAT # 09041
1:10 DILUTION
5,000 ppm Available Chlorine
Active ingredient:
Sodium hypochlorite: 0.525%
Other ingredients: 99.475%
Total: 100.000%

KEEP OUT OF REACH OF CHILDREN

CAUTION

FIRST AID

If in eyes:	<ul style="list-style-type: none">Hold eye open and rinse slowly and gently with water for 15 – 20 minutes.Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
If on skin or clothing:	<ul style="list-style-type: none">Take off contaminated clothing.Rinse skin immediately with plenty of water for 15 – 20 minutes.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. In an emergency, call a doctor or Poison Control Center at 1-800-222-1222

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION Harmful if absorbed through skin. Liquid may cause moderate eye irritation. Avoid contact with skin, eyes or clothing. Wear gloves for sensitive skin or prolonged use since it may cause skin irritation. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

**Tuberculocidal and Bactericidal
1 minute minimum contact time
Maximum surface area that can be
disinfected per towelette is 3' x 3'**

ALCAVIS BLEACH-WIPE

PREMOISTENED 8" x 10" TOWELETTE

Suggested areas of usage:

Medical, Dental and Laboratory Counters, Exam Tables, Carts, Point of Care Equipment, Telephones, Sink Tops.

Directions for Use:

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

- Always use personal protective equipment (gloves).
- Open wipe packet.
- Remove premoistened towelette.
- Apply towelette to hard non-porous surface to be disinfected.
- Thoroughly wet surface making sure treated surface remains visibly wet for 1 full minute.
- Use additional wipe if needed to assure continuous 1 minute wet contact time.
- Allow surface area to air dry and discard used towelette.
- Do not re-use used towelette.

Do not use this product as a terminal sterilant/high level disinfectant on any surface or instrument that is introduced directly into human body, either into or in contact with the bloodstream or normally sterile areas of the body, or contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Bleach-Wipe is a 1:10 dilution of 5.25% sodium hypochlorite solution with filtered water to comply with recommended infection control protocol.

STORAGE AND DISPOSAL:

Do not contaminate water, food, or feed by storage or disposal.

Storage: Store this product in a cool, dry and well-ventilated area away from direct sunlight and heat. **Towelette Disposal:** Do not reuse towelette.

Dispose of used towelette and empty pack in trash. Do not flush in toilet.



Manufactured by:

Angelini Pharma Inc.
8322 Helgerman Court
Gaithersburg, MD 20877
1 (800) 726-2308

Contents of Case

8 Boxes per Case
100 Packets in Box
1 Towel per Packet

Net Contents of
Case: 55 lbs.