



STATE OF HAWAII  
Department of Agriculture

**ACCEPTED**

LICENSE NO.

**7015.3**

EPA Est # 65787-MD-1

EPA Reg No. 65787-4

2 PREMOISTENED 8" x 10" TOWELETTES



**KEEP OUT OF REACH OF CHILDREN  
CAUTION**

**FIRST AID**

If in eyes:	<ul style="list-style-type: none"> <li>• Hold eye open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.</li> <li>• Call a poison control center or doctor for treatment advice.</li> </ul>
<p>Have the product container or label with you when calling a poison control center or doctor, or going for treatment. Contact the Poison Control Center at 1-800-222-1222 for 24 hour emergency medical treatment information.</p>	

**Precautionary Statements.**

**Hazards to Humans and Domestic Animals.**

CAUTION: Causes minimal eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

**CAT # 09032**

**1:100 DILUTION**

**1:100 Dilution of Sodium Hypochlorite Bleach  
600 ppm Available Chlorine**

Active ingredient:  
Sodium hypochlorite: 00.075%  
Other ingredients: 99.925%  
Total: 100.000%

**ALCAVIS BLEACH-WIPE 1:100**

Wipe has 600 ppm available chlorine

**Net Contents: 1.9 oz.**

**BACTERICIDAL**  
Effective in the presence of a  
5% organic soil load against:

*Staphylococcus aureus*  
*Pseudomonas aeruginosa*  
*Salmonella enterica*

**KEEP OUT OF REACH OF CHILDREN**

**Areas of Use:**

For use on hard, nonporous surfaces including stainless steel, chrome, metal, glass, vinyl, plastic, laminate, linoleum, glazed ceramic tile, glazed porcelain, and sealed or painted wood. Areas of use include: Medical, Dental and Laboratory Counters, Exam Tables, Carts, Dialysis Machines, Point of Care Equipment, Telephones, Sink Tops, Sinks, Counter Tops, Floors, Bathrooms, Tubs, Handrails, Door Knobs, Lockers, and Desks.

**Directions for Use:**

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

- Use personal protective equipment (gloves) when disinfecting biohazardous surfaces.
- For heavily soiled surfaces, a preliminary cleaning is required.
- Open wipe packet.
- Remove premoistened towelette.
- Apply towelette and wipe desired surface to be disinfected.
- Treated surface must remain visibly wet for 5 minutes.
- Use additional wipe(s) if needed to assure continuous 5 minute wet contact time.
- Allow surface area to dry and discard used towelette.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

**STORAGE AND DISPOSAL:**

Do not contaminate water, food, or feed by storage and disposal.

**Storage:** Do not store near heat or open flame.

**Towelette Disposal:** Do not reuse towelette. Dispose of used towelette and empty packet in trash after use. Do not flush in toilet.



Manufactured by:  
Angelini Pharma Inc.  
8322 Helgerman Court  
Gaithersburg, MD 20877  
1 (800) 726-2308