



STATE OF HAWAII
Department of Agriculture
ACCEPTED
LICENSE NO. **8013.85**



HEALTHCARE

**Easy on
Surfaces**

Disinfecting Wipes

For Medical + Dental Professional Use



Does Not Contain Bleach

ACTIVE INGREDIENTS:
 n-Alkyl (C14, 60%; C16, 30%; C12, 5%; C18, 5%)
 Dimethyl Benzyl Ammonium Chloride.....0.145%
 n-Alkyl (C12, 68%; C14, 32%)
 Dimethyl Ethylbenzyl Ammonium Chloride..0.145%
 OTHER INGREDIENTS.....99.710%
 TOTAL.....100.000%

KEEP OUT OF REACH OF CHILDREN
CAUTION: SEE BACK PANEL FOR ADDITIONAL
 PRECAUTIONARY STATEMENTS.

75 PREMOISTENED WIPES
NET WT 1.29 LB (587 g)



To Open Package: Open pouch by peeling back seal. When finished, close to retain moisture. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semicritical medical devices prior to sterilization or high-level disinfection.

• This product has demonstrated effectiveness against influenza A2 virus and is expected to inactivate all influenza A viruses, including Pandemic 2009 H1N1 influenza A virus.

PRECAUTIONARY STATEMENTS: HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Wear gloves for prolonged or frequent use.

FIRST AID: IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for further treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. **QUESTIONS?** Call 1-888-797-7225.


PHYSICAL OR CHEMICAL HAZARDS: Do not use or store near heat or open flame.

STORAGE AND DISPOSAL:

Dispose of wipe in trash after use. Do not flush. Reseal pouch between uses to retain moisture. Recycle empty pouch or discard in trash. CONTAINS NO PHOSPHORUS.

QUESTIONS OR COMMENTS? Call 1-888-797-7225.

For more product ingredient information, visit www.ingredientinside.com. For SDS, go to: www.cloroxprofessional.com. **Mfd. for Clorox Professional Products Company, 1221 Broadway, Oakland, CA 94612.** © 2001, 2015 Made in the U.S.A. Clorox Healthcare and Clorox are registered trademarks of The Clorox Company. EPA Reg. No. 67619-9, EPA Est. No. 8251-WI-3(KV), WI-5(UQ), 58455-IN-1(C6), 56952-WI-1(RK), 5813-6A-2(VG), 50757-WI-1(VW). Beginning of batch code indicates Est. No.

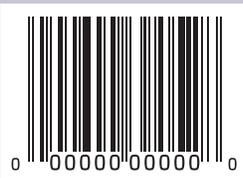

PEEL HERE

KEEP OUT OF REACH OF CHILDREN
CAUTION: SEE BACK PANEL FOR ADDITIONAL PRECAUTIONARY STATEMENTS.

9 PREMOISTENED WIPES NET WT 2.3 OZ (64 g)

ACTIVE INGREDIENTS:
 n-Alkyl (C14, 60%; C16, 30%; C12, 5%; C18, 5%)
 Dimethyl Benzyl Ammonium Chloride 0.145%
 n-Alkyl (C12, 68%; C14, 32%)
 Dimethyl Ethylbenzyl Ammonium Chloride... 0.145%
OTHER INGREDIENTS: 99.710%
TOTAL: 100.000%

DIRECTIONS FOR USE: It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **Not for cleaning or sanitizing skin. Do not use as a diaper wipe or for personal cleansing. For surfaces that may come in contact with food, a potable water rinse is required. Do not use on glasses, dishes or utensils.** These wipes will not harm most surfaces, including acrylic, fiberglass and vinyl. Not recommended for use on unpainted wood. For surfaces that incorporate electronic components, please consult manufacturer's instructions.

TO DISINFECT: Use to disinfect hard, nonporous surfaces. Wipe surface to be disinfected. Use enough wipes for treated surface to remain visibly wet for 4 minutes. Let surface dry. For highly soiled surfaces, clean excess dirt first. **Organisms:** Avian Influenza A (H7N9) virus, Avian Influenza A (H3N2) virus, *Bordetella pertussis*, *Campylobacter jejuni*, Duck Hepatitis B virus (HBV), *Enterobacter aerogenes*, *Escherichia coli* O157:H7 (*E. coli*), Herpes simplex virus type 1, Human immunodeficiency virus type 1 (HIV-1), Human Coronavirus, Influenza A2 Virus, *Klebsiella pneumoniae*, *Listeria monocytogenes* (*Listeria*), Methicillin-resistant *Staphylococcus aureus* (MRSA), *Pseudomonas aeruginosa*, Rotavirus WA, Respiratory syncytial virus (RSV), *Salmonella enterica* (*Salmonella*), *Staphylococcus aureus*, *Streptococcus pyogenes*.

Special Instructions for Use Against HIV-1 and HBV: This product kills HIV-1 and HBV on pre-cleaned hard, nonporous surfaces/objects previously soiled with blood/body fluids in health care settings (hospitals, nursing homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus (HIV-1) (associated with AIDS) and Hepatitis B Virus (HBV).

Special instructions for using this product to clean and decontaminate against HIV-1 and HBV on surfaces/objects soiled with blood/body fluids: Personal Protection: When handling items soiled with blood or body fluids, use disposable impervious gloves, gowns, masks and eye coverings. **Cleaning Procedure:** Blood and other body fluids must be thoroughly cleaned from surfaces and other objects before applying this product. **Contact Time:** Allow surface to remain wet 1 minute for HIV-1, 4 minutes for HBV. **Disposal of Infectious Materials:** Use disposable impervious gloves, gowns, masks and eye coverings. Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

*Kills Avian Influenza A virus on pre-cleaned hard, nonporous surfaces.