



LICENSED

PERIOD 2015-2017 LIC. NO. 8031.2

HENRY SCHEIN®

VAPO-SURE® SOLUTION

is a sterilizing compound for use in
Harvey Chemiclave sterilizers equipped with internal
formaldehyde-filtering purge or with external Chemipurge kit.
To be used for sterilizing dental and surgical instruments.

ACTIVE INGREDIENTS:	Ethanol	72.36%
	Formaldehyde	0.23%
INERT INGREDIENTS		27.36%
TOTAL		100.00%

Contains Methanol

DANGER

See other precautionary statements on back panel

KEEP OUT OF REACH OF CHILDREN

NET CONTENTS

33.8 FL. OZ. (1.0 liter) U.S. MEASURE

Distributed by:
Henry Schein Inc.
Melville, NY 11747 USA



Department of Agriculture
STATE OF HAWAII

LICENSED

PERIOD 2006-2008 LIC. NO.



Department of Agriculture
STATE OF HAWAII

LICENSED

PERIOD 2009-2011 LIC. NO.

Outside line does not print, for size reference only

2003-2007

LICENSED
HAWAII
DEPT. OF AGRICULTURE
2000-2002 8031.2

PRECAUTIONARY STATEMENTS HAZARDS TO HUMANS

DANGER

Causes eye damage and may cause skin irritation. Do not get in eyes, or on skin or on clothing. Wear goggles or face shield when handling. Harmful or fatal if swallowed. Avoid contamination of food.

STATEMENT OF PRACTICAL TREATMENT

In case of contact, immediately flush eyes or skin with plenty of water for at least 15 minutes. For eyes, call a physician. Get medical attention if irritation persists.

NOTE TO PHYSICIAN

Probable mucosal damage may contraindicate the use of gastric lavage.

PHYSICAL AND CHEMICAL HAZARDS

FLAMMABLE. Keep away from heat or open flame. VOLATILE SOLVENTS. Do not use Vapo-Sure in steam sterilizers.

DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Use quantity of undiluted Vapo-Sure sufficient to fill reservoir tank. Refer to operating instructions accompanying each Harvey Chemiclave.

This compound is intended for use in Harvey Chemiclave sterilizers equipped with provisions for mitigating worker exposure to formaldehyde. Harvey EM and EC Series and 7000 and 8000 models have internal filtered purge. Other Harvey Chemiclave sterilizers must be fitted with external Chemipurge kits.

STORAGE AND DISPOSAL

Store in an upright position in a cool dry location. Do not reuse Vapo-Sure. Contact your state and local authorities for proper disposal procedures.

CONTAINER DISPOSAL

Do not reuse container. Triple rinse (or equivalent) container. Then offer container for recycling or reconditioning, or puncture and dispose of it in a sanitary landfill, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

