

4.25"

Special Instructions for Cleaning and Decontamination Against HIV-1 and HBV of Surfaces or Objects Soiled With Blood or Body Fluids:

Personal Protection: When using this disinfecting towelette, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.

Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A PD Care Wipes towelette can be used for this purpose.

Contact Time: Thoroughly wet surface with a new PD Care Wipes towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.

Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids should be autoclaved and disposed of in accordance with Federal, State and local regulations for infectious materials disposal.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

CAUTION

Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for further advice.

If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment.

For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300. For ALL other inquiries about this product, call Customer Service at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL

Towelette - Do not reuse towelette. Dispose of used towelette in trash.

Dispenser - Do not reuse empty container. Wrap container and discard in trash.

Distributed by:



Corporate Office
1031 Mendota Heights Rd.
Saint Paul, MN 55120



+D66207087720920

70-7511-9 • 80092-226



LICENSED

PERIOD 2009-2011 LIC. NO.



LICENSED

PERIOD 2012-2014 LIC. NO.



LICENSED

PERIOD 2015-2017 LIC. NO.

8298.1



For Professional Use

- TUBERCULOCIDAL • BACTERICIDAL
- VIRUCIDAL* • FUNGICIDAL

pd CARE Wipes™

Disinfecting Towelettes

KEEP OUT OF REACH OF CHILDREN CAUTION

See other panel for additional precautionary statements.

PD CARE Wipes Disinfecting Towelettes are non-woven disposable towelettes intended for use on hard, non-porous surfaces in health care settings such as dental offices, hospitals, laboratories and clinics. PD Care Wipes are effective against the following microorganisms when used as directed:

- *Mycobacterium tuberculosis* var: *bovis* (BCG)
- *Staphylococcus aureus*
- *Pseudomonas aeruginosa*
- *Salmonella choleraesuis*
- *Trichophyton mentagrophytes*
- Methicillin Resistant *Staphylococcus aureus* (MRSA)
- Vancomycin Resistant *Enterococcus faecalis* (VRE)
- Hepatitis B Virus (HBV)*
- Herpes Simplex Virus Types 1 and 2*
- Human Immunodeficiency Virus (HIV-1)*

ACTIVE INGREDIENTS:	
Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride0.28%
Isopropanol17.20%
INERT INGREDIENTS†:82.52%
TOTAL100.00%

† Does not include the weight of the dry wipe.

Net Contents: 1 lb. 13 oz.
160 Pre-Saturated Towelettes (6" x 6.75")


Reorder Number
087-7209

EPA Reg. No. 46781-8-43100 • EPA Est. No. 56952-WI-001

MADE IN U.S.A.

16.75"

- Critical Text Box
- - - - - Bleeds
- - - - - Adhesive area
- Die Cut area

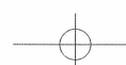

DIREC
It is a
manne
PD Ca
porous
offices,
center
care ar
cross
require
PD Ca
directe
operat
amalg
tables
exteri
respira
count
endot
testers
knobs,
toilet
cabine
polycy
polysty
of non
Plexigl
DO NC
Disper
To sta
centro
a contain



For Professional Use
**IDAL • BACTERICIDAL
 FUNGICIDAL**

**KEEP OUT OF REACH
 OF CHILDREN
 CAUTION**

Other panel for additional
 cautionary statements.

For use on hard, non-porous
 surfaces. PD Care Wipes are effective

ethyl benzyl ammonium chloride0.28%
17.20%
 **82.52%**
100.00%
 dry wipe.

13 oz.
 x 6.75"

Reorder Number

087-7209

S.A.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

PD Care Wipes are intended for use on hard, non-porous surfaces in health care settings such as dental offices, hospitals, emergency medical settings, surgical centers, out-patient surgical centers (OPSC), patient care areas, and other critical care areas where control of cross contamination between treated surfaces is required.

PD Care Wipes will clean and disinfect, when used as directed, such items as dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, operating room tables and lights, laboratory equipment and surfaces, exterior surfaces of anesthesia machines and respiratory therapy equipment, workstations, dental countertops, dental chairs, loupes, exterior surfaces of endodontic equipment, such as apex locators, pulp testers and motors, stools, floors, walls, handrails, door knobs, tables, chairs, faucets, bathroom fixtures, exterior toilet surfaces, garbage cans, computers, keyboards, cabinets and telephones, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

DO NOT USE AS A BABY WIPE. Not for use on skin.

Dispenser Directions:

To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace

cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

Cleaning Instructions:

Use one PD Care Wipes towelette to completely preclean surface of all gross debris.

For use as a virucide*, including HIV-1 and HBV applications and effectiveness against Methicillin Resistant *Staphylococcus aureus* (MRSA) and Vancomycin Resistant *Enterococcus faecalis* (VRE):

Use a second PD Care Wipes towelette to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes at room temperature (69°F/20°C).

For use as a disinfectant:

Use a second PD Care Wipes towelette to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at room temperature (69°F/20°C) for Tuberculocidal activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella choleraesuis*.

PD Care Wipes effectively kills Human Immunodeficiency Virus Type 1 (HIV) and Hepatitis B Virus (HBV) on precleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of HIV and HBV.

.75"

No Print Area • Glue Adhesive

