

Based on EPA stamped label dated 6-9-15

Use GymWipes Antibacterial Fast 4 wipes to clean and disinfect hard non-porous surfaces of gym equipment, exercise machines, wrestling mats and non-critical medical equipment surfaces: examining tables. For use in hospitals, schools, health clubs and daycare centers.

TO OPEN PACKAGE: Slowly open seal. Remove towelette as needed. To retain moisture, reseal bag by firmly pressing seal back on the bag.

DIRECTIONS FOR USE:

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

Do not use as a diaper wipe or for personal cleansing. This is not a baby wipe! Do not flush in toilet. For plastic and painted surfaces, spot test on an inconspicuous area before use. Will not harm most surfaces: including acrylic, sealed fiberglass and vinyl. Do not use on unpainted wood.

TO CLEAN AND DEODORIZE: Wipe surface with wipe. Let air dry. Discard used wipe in trash. Do not flush in toilet.

TO CLEAN AND DISINFECT / VIRUCIDAL* / MOLD AND MILDEW CONTROL DIRECTIONS:

Wipe surface with wipe until surface is visibly wet. Allow surface to remain wet for 4 minutes. Let air dry. If surfaces are extremely dirty, clean first with another wipe before disinfecting. Discard used wipe in trash. Do not flush. Rinse food contact surfaces with clean, potable water after disinfecting. Do not use to disinfect dishes, glassware or utensils.

For mold and mildew, GymWipes Antibacterial Fast 4 wipes will effectively inhibit the growth of mold and mildew and the odors caused by them when applied to hard, nonporous surfaces. Follow disinfection instructions. Repeat treatment every seven days, or more if new growth appears.

***KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES / OBJECTS PREVIOUSLY SOILED WITH BLOOD / BODY FLUIDS**

in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces / objects with blood or body fluids, and in which the surfaces / objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 [HIV-1] (associated with AIDS).

DIRECTIONS CONTINUED ON RIGHT PANEL

GYMWIPES[®]

ANTIBACTERIAL

FAST 4

Fast Drying, Antibacterial Formula, Cleans, Disinfects and Deodorizes

Kills 99.9% of Bacteria in Just 15 Seconds!

Effective against: *Staphylococcus aureus* - Methicillin-Resistant [MRSA] *Staphylococcus aureus* - Community Associated Methicillin-Resistant [CA-MRSA] [NRS384] [USA300], *Pseudomonas aeruginosa*, *Salmonella enterica*, *Staphylococcus aureus* (Staph), *HIV-1 [AIDS Virus], *Avian Influenza Virus H5N1

- Virucide*
- Ammonia-Free
- Bleach Free

ACTIVE INGREDIENTS:

Octyl decyl dimethyl ammonium chloride.	0.0909%
Dioctyl dimethyl ammonium chloride.	0.0364%
Didecyl dimethyl ammonium chloride.	0.0545%
Alkyl (C ₁₄ , 50%; C ₁₂ , 40%; C ₁₆ , 10%) dimethyl benzyl ammonium chloride.	0.1212%
Other Ingredients:	99.6970%
Total	100.0000% ¹

¹Includes weight of absorbent wipe.



700
LARGE 8" x 6"
TOWELETTES
NET CONTENTS
10 lbs. 11 oz.

See right panel for Precautionary Statements and First Aid
KEEP OUT OF REACH OF CHILDREN
CAUTION



CONTINUED DIRECTIONS FOR USE

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES / OBJECTS SOILED WITH BLOOD / BODY FLUIDS.

Personal protection: Clean-up must always be done wearing protective gloves, gowns, masks and eye protection.

Cleaning procedure: Blood and other body fluids containing HIV must be thoroughly cleaned from surfaces and objects before application of GymWipes Antibacterial Fast 4 wipes. GymWipes Antibacterial Fast 4 wipes can be used for this purpose.

Contact time: Leave surface wet for 1 minute [60 seconds] for HIV-1. Use appropriate contact time as listed on label for all other viruses and bacteria claimed.

Disposal of infectious material:

Blood, body fluids, cleaning materials and clothing must be autoclaved and disposed of according to local regulations for infectious waste disposal.

This product is for use on hard non-porous surfaces not conducive to treatment by immersion or excess liquid against Avian Influenza A.

PRECAUTIONARY STATEMENTS

HAZARDS TO HUMANS AND DOMESTIC ANIMALS

Caution. Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then continue rinsing eye.

Call a poison control center or doctor for treatment advice. Have the product container or label with you when calling poison control center or doctor or going for treatment.

STORAGE AND DISPOSAL

Store in original container in areas inaccessible to children. Keep securely closed. Refillable container. Refill empty container only with GymWipes Antibacterial Fast 4 wipes refills. Do not reuse this container except as described. If not refilling, discard empty container in trash or recycle.

EPA REG. NO. 6836-336-82144
EPA EST. NO. 8251-WI-05

STATE OF HAWAII
Department of Agriculture

ACCEPTED
LICENSE NO. **8456.8**



2XL-101 REFILL



2XL Corporation
7550 Industrial Drive
Forest Park IL 60130-2516
(888) 977-3726
www.2xlcorp.com

Sold by: 2XL Corporation