



STATE OF HAWAII
Department of Agriculture

ACCEPTED

LICENSE NO.

8586.1

BluTab®

Waterline Maintenance Tablets

Dental Unit Waterline Treatment

Inhibits the Growth of Microbial Contamination in Dental Unit Waterlines

ACTIVE INGREDIENT

Silver	0.477%
OTHER INGREDIENTS	99.523%
TOTAL	100.000%

CAUTION

KEEP OUT OF REACH OF CHILDREN
SEE BACK PANEL FOR ADDITIONAL
PRECAUTIONARY STATEMENTS

Net Contents: 50 tablets (0.26 oz. total)

www.blutab.com

Directions for Use:

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING. READ ENTIRE LABEL BEFORE USING THIS PRODUCT.

Initial Treatment

Before starting a treatment program with BluTab® Waterline Maintenance Tablets, flush lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When flushing dental unit water lines, always follow a recognized line flushing/cleaning protocol. Repeat flushing to clean lines every 2 months or more frequently when water quality results indicate.

Routine Treatment

1. Place tablet or tablets (see table below) in empty dental unit water bottle every time you refill the water. BluTab® Waterline Maintenance Tablets are intended for use with potable water (including tap or distilled).
2. Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.
3. BluTab® Waterline Maintenance Tablets may be left in the waterlines overnight and for extended periods of non-use. Use water solution until it is expended or up to 28 days. It is not necessary to purge lines at the end of each day.

Water Volume	Tablets
700-750 ml (24-25 fl. oz.)	1
1500 ml (51 fl. oz.)	2
2000 ml (68 fl. oz.)	3



Clinical monitoring of water quality can ensure that procedures are being performed correctly and are effective. Test water quality monthly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.

Precautionary Statements: Hazards to Humans and Domestic Animals.

CAUTION: Tablet may cause moderate eye irritation. Avoid contacting tablet with eyes or clothing. Wash thoroughly with soap and water after handling tablets and before eating, drinking, chewing gum, or using tobacco.

First Aid: If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first five minutes and continue rinsing. Call a poison control center or doctor for treatment advice.

Hotline Number: In case of emergency call toll free 1-866-290-1830. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

BluTab® Waterline Maintenance Tablets are designed to be continuously present in your dental unit water system.

Storage and Disposal

Storage: Do not contaminate water, food or feed by storage or disposal. Store in a cool dry place.

FOIL POUCH

Disposal: If empty: Place in trash or offer for recycling if available. If partly filled: Call your solid waste agency (or 1-800-CLEANUP) for disposal instructions.

OUTER CONTAINER

Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of the empty container in trash.

Distributed by: ProEdge Dental Products
Centennial, CO 80112



7348 S. Alton Way, Unit 9D
Centennial, CO 80112

EPA Reg. No. 84020-1

EPA Establishment No. 84065-MO-001

REORDER NO.: BT50

1-888-843-3343



Clinical monitoring of water quality can ensure that procedures are being performed correctly and are effective. Test water quality monthly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.

Precautionary Statements: Hazards to Humans and Domestic Animals.

CAUTION: Tablet may cause moderate eye irritation. Avoid contacting tablet with eyes or clothing. Wash thoroughly with soap and water after handling tablets and before eating, drinking, chewing gum, or using tobacco.

First Aid: If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first five minutes and continue rinsing. Call a poison control center or doctor for treatment advice.

Hotline Number: In case of emergency call toll free 1-866-290-1830. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

BluTab® Waterline Maintenance Tablets are designed to be continuously present in your dental unit water system.

REORDER NO.: BT50

1-888-843-3343

Storage and Disposal

Storage: Do not contaminate water, food or feed by storage or disposal. Store in a cool dry place.

FOIL POUCH

Disposal: If empty: Place in trash or offer for recycling if available. If partly filled: Call your solid waste agency (or 1-800-CLEANUP) for disposal instructions.

OUTER CONTAINER

Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of the empty container in trash.

Distributed by: ProEdge Dental Products
Centennial, CO 80112



7348 S. Alton Way, Unit 9D
Centennial, CO 80112

EPA Reg. No. 84020-1

EPA Establishment No. 84065-MO-001

BluTab®

Waterline Maintenance Tablets

Dental Unit Waterline Treatment

Inhibits the Growth of Microbial Contamination in Dental Unit Waterlines

ACTIVE INGREDIENT

Silver 0.477%
OTHER INGREDIENTS 99.523%
TOTAL 100.000%

CAUTION

KEEP OUT OF REACH OF CHILDREN
SEE BACK PANEL FOR ADDITIONAL
PRECAUTIONARY STATEMENTS

Net Contents: 50 tablets (0.26 oz. total)

www.blutab.com

Directions for Use:

IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING. READ ENTIRE LABEL BEFORE USING THIS PRODUCT.

Initial Treatment

Before starting a treatment program with BluTab® Waterline Maintenance Tablets, flush lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When flushing dental unit water lines, always follow a recognized line flushing/cleaning protocol. Repeat flushing to clean lines every 2 months or more frequently when water quality results indicate.

Routine Treatment

- Place tablet or tablets (see table below) in empty dental unit water bottle every time you refill the water. BluTab® Waterline Maintenance Tablets are intended for use with potable water (including tap or distilled).
- Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.
- BluTab® Waterline Maintenance Tablets may be left in the waterlines overnight and for extended periods of non-use. Use water solution until it is expended or up to 28 days. It is not necessary to purge lines at the end of each day.

Water Volume	Tablets
700-750 ml (24-25 fl. oz.)	1
1500 ml (51 fl. oz.)	2
2000 ml (68 fl. oz.)	3