



LICENSED

PERIOD 2014-2016 LIC. NO.

8623.1  
Disc.



LICENSED

PERIOD 2011-2013 LIC. NO.

Premoistened 3" x 3" Towel

REORDER REF 0 3757897001

TUBERCULOCIDAL VIRUCIDAL\* BACTERICIDAL FUNGICIDAL

## ACCU-CHEK Convenience Bleach Wipe

Disinfecting wipe for hard, non-porous environmental surfaces and patient care equipment effective against *Mycobacterium bovis variant tuberculosis* OT 451C150, \*HIV-1, *Trichophyton mentagrophytes*, *Staphylococcus aureus*-MRSA, *Enterococcus faecalis*-VRE, *Staphylococcus aureus*, *Salmonella enterica* and *Pseudomonas aeruginosa*

For wiping hard non-porous environmental surfaces & patient care equipment

Active ingredient: Sodium hypochlorite.....	0.525%
Other ingredients* .....	99.475%
Total.....	100.000%

\*Other ingredients do not include weight of towel.

SUGGESTED AREAS OF USAGE: Medical, Dental and Laboratory Counters, Exam Tables, Carts, Patient Care Equipment and Point-of-Care Equipment

KEEP OUT OF REACH OF CHILDREN

### CAUTION

FIRST AID: If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

#### STORAGE AND DISPOSAL

- Store product in a cool, dry area, away from heat and direct sunlight.
- Do not reuse empty packet and used towel
- Discard used towel and empty packet in designated infectious waste container for autoclaving and disposal according to federal, state, and local regulations.

(See panel on reverse side for directions for use and additional precautionary statements.)

EPA Reg. No. 69687-1-75446 EPA Est. No. 69687-IL-001

Net Contents 0.1 oz.

- For product questions, call ACCU-CHEK® Customer Care Service Center at 1-800-440-3638.
- Complete product line information is available at [www.accu-chek.com](http://www.accu-chek.com).

**Directions for use:**

- It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.
- Always use personal protective equipment.
- Gross filth and heavy soil should be removed prior to disinfection.
- Open ACCU-CHEK Convenience Bleach Wipe packet.
- Remove premoistened 3" x 3" towel.
- Apply towel and wipe desired surface to be disinfected.
- A 5 minute contact is required for effectiveness against organisms listed.
- Allow surface to air dry and discard used towel and empty packet (see front panel).

**\*KILLS HIV-1 ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS**

in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids and in which the surfaces/objects likely to be soiled with blood/body fluids can be associated with the potential for transmission of HIV-1 (associated with AIDS). SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS. • PERSONAL PROTECTION: When handling items soiled with blood or body fluids, use disposable gloves, gown, mask and eye protection. • CLEANING PROCEDURES: Blood/body fluids must be thoroughly cleaned from surfaces/objects before application of ACCU-CHEK Convenience Bleach Towel. • CONTACT TIME: Allow surface to remain wet for 5 minutes. • DISPOSAL OF INFECTIOUS MATERIAL: Blood/body fluids should be autoclaved and disposed of according to local regulations for infectious waste disposal.

**Precautionary Statements:** Hazards to humans and domestic animals.

**CAUTION:** Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling. This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barriers or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

Assembled for and distributed in the U.S.A. by:  
Roche Diagnostics Corporation, 9115 Hague Road, Indianapolis, IN 46256

Made in the U.S.A. ACCU-CHEK is a trademark of Roche.

© 2010, Roche Diagnostics 28196451005-0210

