

Client: _____
 Job #: _____ Date: 03-03/08
 Item: Mydent International
 Size: 3.68" X 6"

PLEASE CHECK YOUR PROOF CAREFULLY!

Please review this proof carefully. Please check names, numbers, spelling, punctuation, layout, size, etc. We are asking that you take the responsibility of final approval before we go to press. Thank you for your assistance!

Before your order can be printed, this proof must be returned to _____ with your signed approval.

- Proof is OK. Print as is.
- Minor correction, OK to print after change.
- Changes needed as indicated, please proof again.

Signature: _____ Date: _____ Job #: _____

This Proof Indicates Approximate Color Only.
 For Accurate Color Match, Use Actual PANTONE Chips.
 DO NOT USE COLORS ON THIS FILE AS THEY DO NOT MATCH PMS COLORS.



*Use PMS #294C



FINAL UNWIND

LICENSED

LICENSED

DEFEND PLUS
 by Mydent International, Inc.

Item #SO-9000

Disinfecting/Deodorizing/Cleaning Wipes
 FOR USE ON ENVIRONMENTAL SURFACES

A nonwoven disposable cloth containing a stable, low pH formulated disinfectant and deodorant for use on hard, non-porous surfaces, in hospitals, intensive care units, surgery, X-ray, Cat. Lab., orthopedics, newborn nursery, emergency medical settings, laboratories, clinics, nursing facilities, medical offices, dental offices, veterinary facilities - where control of cross-contamination is required. For use on hard non-porous surfaces only, such as stainless steel, Formica, glass tables, carts, baskets, counters, cabinets, telephones.

PREPARATION OF DISPENSER: Remove cover from canister and discard covering seal. From center of wipes roll, pull up a wipe corner, twist into a point and thread through the X hole in the underside of the lid cover. Pull wipe through about one (1) inch. Replace the lid cover on container, assuring a secure fit. Pull out first wipe and snap at a 90° angle. Remaining wipes feed automatically for next use. Always reseal hinged cap after each use to insure moisture retention.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes, but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

ACTIVE INGREDIENTS:			
Virucidal*	N-Alkyl (60% C14, 30% C16, 5% C12, 5% C18)	0.12%	
	Dimethyl Benzyl Ammonium Chloride		
Bactericidal	N-Alkyl (68% C12, 32% C14)	0.12%	
	Dimethyl Ethyl Ammonium Chloride		
Pseudomonocidal	Isopropyl Alcohol	41.58%	
Tuberculocidal			
Fungicidal			
INERT INGREDIENTS+:		58.15%	
TOTAL		100.00%	

KEEP OUT OF REACH OF CHILDREN
WARNING
DO NOT USE AS A BABY WIPE
 See back panel for additional precautionary statements.

Net Contents: **160** Presaturated Wipes (6 x 6.75 in.)
 (Chemtrec 24hr Emergency Tel# 1-800-424-9300)



LICENSED

FINAL UNWIND

DEFEND PLUS DISINFECTING/DEODORIZING/CLEANING WIPES by Mydent International

DIRECTIONS FOR USE: It is a violation of U.S. Federal Law to use this Product in a manner inconsistent with its labeling.
DISINFECTION: Thoroughly wet pre-cleaned, hard, non-porous surface with a wipe, keep wet for 5 minutes, and allow to air dry. For use as a general disinfectant wipe on hard, nonporous, inanimate surfaces such as chairs, counter tops, exam tables, office furniture, chart racks and equipment, cabinets, door knobs, sinks, mirrors, light fixtures and switches, supply, and trash carts, telephones, waste cans and laundry hampers, plastic and other similar hard surfaces. You may use a wipe to pre-clean the surface to be disinfected, but thorough rewiping with a fresh wipe is required for disinfection. Not to be used on food contact surfaces.

DEFEND PLUS Disinfecting/Deodorizing/Cleaning Wipes are effective against the following pathogens in 5 minutes at 20° C/68° F, on hard, non-porous surfaces when used as directed: Staphylococcus aureus, Trichophyton mentagrophytes (Athlete's foot fungus), Pseudomonas aeruginosa, Mycobacterium tuberculosis, Salmonella choleraesuis, Methicillin Resistant Staphylococcus aureus (MRSA), Escherichia coli 0157:H7, Vancomycin Resistant Enterococcus faecalis (VRE), *Herpes simplex virus 2 (genital Herpes virus) *Influenza A (A2 Japan) *HIV-1 (AIDS virus) ** (At an exposure time of 1 min.) **See panel for complete HIV-1 (AIDS virus) information.

DEODORIZING: Wipe hard, non-porous surface thoroughly to eliminate odors from mildew and mold.

CLEANING: To clean surfaces, wipe thoroughly with a fresh wipe, which may be used to deodorize at the same time as cleaning. Use in bathrooms, industrial plants, kitchens (non-food use surfaces), sickrooms, musty closets and similar areas. When used to clean a hard non-porous surface prior to disinfection, a second wipe must be used for disinfection.

THE ACTIVITY OF DEFEND PLUS DISINFECTING/DEODORIZING/CLEANING WIPES AGAINST THE HIV-1 (AIDS VIRUS): DEFEND PLUS Disinfecting/Deodorizing/Cleaning Wipes completely inactivate the HIV-1 (AIDS virus) on hard, non-porous surfaces when used at a contact time of 1 minute at room temperature (20-25° C). DEFEND PLUS Disinfecting/Deodorizing/Cleaning Wipes kill HIV-1 on pre-cleaned environmental surfaces/objects soiled with blood/body fluids, in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type-1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS: Personal Protection: Specific barrier protection items to be used when handling items soiled with blood or body fluids are disposable latex gloves, gowns, masks or eye coverings. Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of DEFEND PLUS Disinfecting/Deodorizing/Cleaning Wipes. Contact Time: HIV-1 virus is inactivated in 1 minute, however, other pathogens require 5 minutes exposure for complete kill. See disinfection directions.

Disposal of Infectious Materials: Materials that may contain blood or other body fluids should be autoclaved and disposed of according to federal, state and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS:

HAZARDS TO HUMANS AND DOMESTIC ANIMALS: WARNING: Causes substantial, but temporary eye injury. Do not get in eyes or on clothing. Wear specific appropriate protective eyewear such as goggles, face shield or safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, or using tobacco. In case of eye contact, flush with plenty of water for at least 15 minutes. If irritation persists, call a physician. Wear safety glasses and rubber gloves, as appropriate, when handling HIV-1 infected blood or body fluids. Avoid contamination of food. Remove and wash contaminated clothing before reuse.

FIRST AID: If in eyes, hold eye open and rinse slowly and gently with water for at least 15-20 minutes. Remove contact lenses, if present, after first 5 minutes, then, continue rinsing eye. Call a poison control center or doctor for treatment advice. If on skin or clothing, take off contaminated clothing. Rinse skin immediately with plenty of water for 15 to 20 minutes. Call a poison control center or doctor for treatment advice. Wash contaminated clothing before reuse. WASH THOROUGHLY AFTER HANDLING WIPES.

STORAGE AND DISPOSAL: Store in cool area. Keep away from heat or open flame. Do not reuse wipes. Dispose of used wipe and wrapped empty dispenser in trash. For technical information call 800-275-0020.

Distributed by: **DEFEND PLUS** by Mydent International, Inc.
 80 Suffolk Court • Hauppauge, NY 11787 • www.defend.com
 EPA Reg. No. 59894-10-82666
 EPA Est. No. 73027-WI-01

CAT. NO. SO-9000



1.50" Round Top Cap Label