



LICENSED

PERIOD 2015-2017 LIC. NO.

8869.1



LICENSED

PERIOD 2012-2014 LIC. NO.



Net Contents
2 fl oz
60 mL

Hospital Hard Surface Disinfectant

BioSURF is a ready to use hard surface disinfectant
for nonporous surfaces • One Step

<u>Active Ingredients</u>	
Ethanol	70.5%
Chlorhexidine Gluconate	0.2%
<u>Other Ingredients</u>	29.3%
Total	100.0%

EPA Registration Number 70467-3
EPA Establishment 070467-CAN-001

Tuberculocidal - Canine Parvovirucidal
Pseudomonacidal



Micrylium

PROFESSIONAL DISINFECTION

www.micrylium.com

3309A Filmore • 5000M Dufferin

San Francisco, CA 94123 • Toronto, Canada M3H5T5
1-800-489-8868

CAUTION

READ PRECAUTIONS ON BACK PANEL • KEEP OUT OF THE REACH OF CHILDREN

Aldehyde • NPE • Phenol Free



BioSURF is a ready to use environmental
hard surface disinfectant for nonporous surfaces

Directions for use

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. *BioSURF*™ kills Tuberculosis (*Mycobacterium bovis* BCG) and is Virucidal (*Canine parvovirus* CPV-265) in one minute in the presence of 5% bioburden. *BioSURF*™ kills *Pseudomonas aeruginosa*, *Staphylococcus aureus* and *Salmonella enterica* (formally *choleraesuis*) in 20 seconds in the presence of 10% bioburden. *BioSURF*™ kills *Trichophyton mentagrophytes* in 10 seconds in the presence of 10% bioburden. *BioSURF*™ is to be used on countertops and metal surfaces. May be used on wheelchairs, metal bed frames and exteriors of toilets. Spray all areas with a light mist. Maintain wetness for one minute to kill *Mycobacterium bovis* BCG and *Canine parvovirus* CPV-265. Maintain wetness for 20 seconds to kill bacteria and 10 seconds to kill fungi, then wipe with disposable paper towels or "Le Cloth"™ towelettes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.




Flammable

Manufactured by
Micrylium Laboratories
DOC # ART-SURF-060-01 US

Precautionary Statements • Hazards to Humans and Domestic Animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wear safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco. **PHYSICAL OR CHEMICAL HAZARDS:** Flammable. Keep away from heat or open flame. Keep out of the reach of children. **DO NOT** use on alcohol sensitive materials such as hot plastic casings, acrylic paint, styrene or disposable plastic items. Do not apply this product in or on electrical equipment due to the possibility of shock hazard. **FIRST AID: IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-489-8868 for emergency medical treatment information. **STORAGE & DISPOSAL:** Do not contaminate water, food or feed by storage or disposal. **PESTICIDE STORAGE:** Do not store near open flame, at elevated temperature or in intense light. **PESTICIDE DISPOSAL:** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility. **CONTAINER HANDLING:** Refillable container. Refill this container with *BioSURF* pesticide only. Do not reuse this container for any other purpose. Cleaning the container before final disposal is the responsibility of the person disposing of the container. Cleaning before refilling is the responsibility of the refiller. Rinse thoroughly and dispose of any packaging materials in domestic waste. Then offer for recycling if available or puncture and dispose of in a sanitary landfill or, by incineration or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

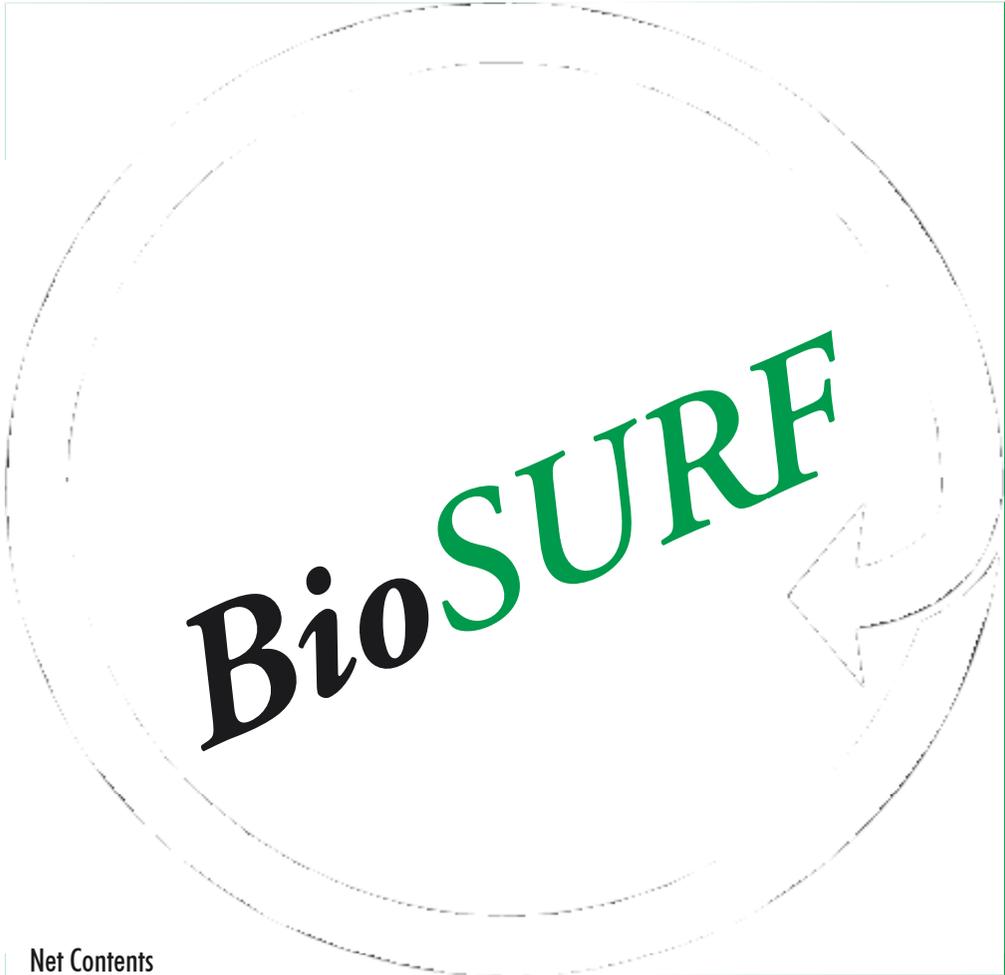

BioSURF

Net Contents

16.9 fl oz
500 ml

EPA Registration Number 70467-3
EPA Establishment 070467-CAN-001

Micrylium
ISO 13485 : 2003

Hospital Hard Surface Disinfectant

BioSURF is a ready to use hard surface disinfectant
for nonporous surfaces • One Step


Active Ingredients

Ethanol	70.5%
Chlorhexidine Gluconate	0.2%
<u>Other Ingredients</u>	29.3%
Total	100.0%

Tuberculocidal - Canine Parvovirucidal
Pseudomonacidal

CAUTION

READ PRECAUTIONS ON BACK PANEL • KEEP OUT OF THE REACH OF CHILDREN

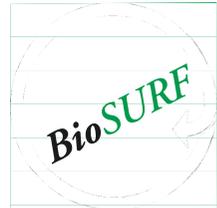

Aldehyde • NPE • Phenol Free

BioSURF is a ready to use environmental hard surface disinfectant for nonporous surfaces

Directions for use

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **BioSURF**™ kills Tuberculosis (*Mycobacterium bovis BCG*) and is Virucidal (*Canine parvovirus CPV-265*) in one minute in the presence of 5% bioburden. **BioSURF** kills *Pseudomonas aeruginosa*, *Staphylococcus aureus* and *Salmonella enterica* (formerly *choleraesuis*) in 20 seconds in the presence of 10% bioburden. **BioSURF**™ kills *Trichophyton mentagrophytes* in 10 seconds in the presence of 10% bioburden. To effectively kill bacteria, fungi and inactivate canine parvovirus use **BioSURF**. Remove trigger from 24 fl oz spray bottle. Cut this 16.9 fl oz refill pouch along the dotted line with scissors. Use a clean funnel to fill or refill bottle. Replace the trigger and rinse funnel. **BioSURF**™ is to be used on countertops and metal surfaces. May be used on wheelchairs, metal bed frames and exteriors of toilets. Spray all areas with a light mist. Maintain wetness for one minute to kill *Mycobacterium bovis BCG* and *Canine parvovirus CPV-265*.

Maintain wetness for 20 seconds to kill bacteria and 10 seconds to kill fungi, then wipe with disposable paper towels or "Le Cloth™" towelettes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.



Flammable



Micrylium

PROFESSIONAL DISINFECTION

www.micrylium.com
3309A Filmore • 5000M Dufferin
San Francisco, CA 94123 • Toronto, Canada M3H5T5
1-800-489-8868

Manufactured by
Micrylium Laboratories

Precautionary Statements Hazards to Humans and Domestic Animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wear safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

PHYSICAL OR CHEMICAL HAZARDS: Flammable. Keep away from heat or open flame. Keep out of the reach of children. DO NOT use on alcohol sensitive materials such as hot plastic casings, acrylic paint, styrene or disposable plastic items. Do not apply this product in or on electrical equipment due to the possibility of shock hazard.

FIRST AID: IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-489-8868 for emergency medical treatment information.

STORAGE & DISPOSAL: Do not contaminate water, food or feed by storage or disposal. **PESTICIDE STORAGE:** Do not store near open flame, at elevated temperature or in intense light. **PESTICIDE DISPOSAL:** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

CONTAINER HANDLING: Nonrefillable container. Do not reuse or refill this container. Then offer for recycling if available or puncture and dispose of in a sanitary landfill or, by incineration or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.



BioSURE

Net Contents
24 fl oz
710 ml

Hospital Hard Surface Disinfectant

BioSURE is a ready to use hard surface disinfectant
for nonporous surfaces • One Step

Active Ingredients

Ethanol	70.5%
Chlorhexidine Gluconate	0.2%
<u>Other Ingredients</u>	29.3%
<u>Total</u>	100.0%

EPA Registration Number 70467-3
EPA Establishment 070467-CAN-001

Tuberculocidal - Canine Parvovirucidal
Pseudomonacidal



Micrylium

PROFESSIONAL DISINFECTION

www.micrylium.com

3309A Filmore • 5000M Dufferin

San Francisco, CA 94123 • Toronto, Canada M3H5T5

1-800-489-8868

CAUTION

READ PRECAUTIONS ON BACK PANEL • KEEP OUT OF THE REACH OF CHILDREN



Aldehyde • NPE • Phenol Free

BioSURF is a ready to use environmental hard surface disinfectant for nonporous surfaces

Directions for use

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **BioSURF**™ kills Tuberculosis (*Mycobacterium bovis BCG*) and is Virucidal (*Canine parvovirus CPV-265*) in one minute in the presence of 5% bioburden. **BioSURF**™ kills *Pseudomonas aeruginosa*, *Staphylococcus aureus* and *Salmonella enterica* (formally *choleraesuis*) in 20 seconds in the presence of 10% bioburden. **BioSURF**™ kills *Trichophyton mentagrophytes* in 10 seconds in the presence of 10% bioburden.

BioSURF™ is to be used on countertops and metal surfaces.

May be used on wheelchairs, metal bed frames and exteriors of toilets. Spray all areas with a light mist. Maintain wetness for one minute to kill *Mycobacterium bovis BCG* and *Canine parvovirus CPV-265*.

Maintain wetness for 20 seconds to kill bacteria and 10 seconds to kill fungi, then wipe with disposable paper towels or "Le Cloth™" towelettes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.



Flammable

Manufactured by
Micrylium Laboratories

DOC # ART-SURF-710-01 US

Precautionary Statements • Hazards to Humans and Domestic Animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wear safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco. **PHYSICAL OR CHEMICAL HAZARDS:** Flammable. Keep away from heat or open flame. Keep out of the reach of children. **DO NOT** use on alcohol sensitive materials such as hot plastic casings, acrylic paint, styrene or disposable plastic items. Do not apply this product in or on electrical equipment due to the possibility of shock hazard. **FIRST AID: IF IN EYES:** Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-489-8868 for emergency medical treatment information. **STORAGE & DISPOSAL:** Do not contaminate water, food or feed by storage or disposal. **PESTICIDE STORAGE:** Do not store near open flame, at elevated temperature or in intense light. **PESTICIDE DISPOSAL:** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility. **CONTAINER HANDLING:** Refillable container. Refill this container with **BioSURF** pesticide only. Do not reuse this container for any other purpose. Cleaning the container before final disposal is the responsibility of the person disposing of the container. Cleaning before refilling is the responsibility of the refiller. Rinse thoroughly and dispose of any packaging materials in domestic waste. Then offer for recycling if available or puncture and dispose of in a sanitary landfill or, by incineration or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

BioSURF

Net Contents

169 fl oz
5000 ml

EPA Registration Number 70467-3
EPA Establishment 070467-CAN-001

Micrylium
ISO 13485 : 2003

Hospital Hard Surface Disinfectant

BioSURF is a ready to use hard surface disinfectant
for nonporous surfaces • One Step

Active Ingredients


Ethanol	70.5%
Chlorhexidine Gluconate	0.2%

<u>Other Ingredients</u>	29.3%
Total	100.0%

Tuberculocidal - Canine Parvovirucidal
Pseudomonacidal

CAUTION

READ PRECAUTIONS ON BACK PANEL • KEEP OUT OF THE REACH OF CHILDREN

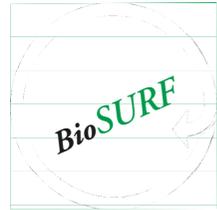

Aldehyde • NPE • Phenol Free

BioSURF is a ready to use environmental hard surface disinfectant for nonporous surfaces


Directions for use

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. **BioSURF**™ kills Tuberculosis (*Mycobacterium bovis BCG*) and is Virucidal (*Canine parvovirus CPV-265*) in one minute in the presence of 5% bioburden. **BioSURF** kills *Pseudomonas aeruginosa*, *Staphylococcus aureus* and *Salmonella enterica* (formerly *choleraesuis*) in 20 seconds in the presence of 10% bioburden. **BioSURF**™ kills *Trichophyton mentagrophytes* in 10 seconds in the presence of 10% bioburden. To effectively kill bacteria, fungi and inactivate canine parvovirus use **BioSURF**. Remove trigger from 24 fl oz spray bottle. Use a clean funnel to fill or refill bottle. Replace the trigger and rinse funnel. **BioSURF**™ is to be used on countertops and metal surfaces. May be used on wheelchairs, metal bed frames and exteriors of toilets. Spray all areas with a light mist. Maintain wetness for one minute to kill *Mycobacterium bovis BCG* and *Canine parvovirus CPV-265*. Maintain wetness for 20 seconds to kill bacteria and 10 seconds to kill fungi, then wipe with disposable paper towels or "Le Cloth™" towelettes.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.



Flammable

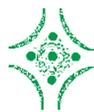

Micrylium

PROFESSIONAL DISINFECTANT

www.micrylium.com
3309A Filmore • 5000M Dufferin
San Francisco, CA 94123 • Toronto, Canada M3H5T5
1-800-489-8868

Manufactured by
Micrylium Laboratories

Precautionary Statements Hazards to Humans and Domestic Animals

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wear safety glasses. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum or using tobacco.

PHYSICAL OR CHEMICAL HAZARDS: Flammable. Keep away from heat or open flame. Keep out of the reach of children. **DO NOT** use on alcohol sensitive materials such as hot plastic casings, acrylic paint, styrene or disposable plastic items. Do not apply this product in or on electrical equipment due to the possibility of shock hazard.

FIRST AID: IF IN EYES: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a Poison Control Center or doctor for treatment advice. Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact 1-800-489-8868 for emergency medical treatment information.

STORAGE & DISPOSAL: Do not contaminate water, food or feed by storage or disposal. **PESTICIDE STORAGE:** Do not store near open flame, at elevated temperature or in intense light. **PESTICIDE DISPOSAL:** Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

CONTAINER HANDLING: Nonrefillable container. Do not reuse or refill this container. Then offer for recycling if available or puncture and dispose of in a sanitary landfill or, by incineration or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.