



LICENSED

PERIOD 2015-2017 LIC. NO. 8877.1

14.875"

Kills C DIFF Spores in ONE STEP in 3 minutes • Kills +HBV & +HCV in 30 seconds • CAN BE USED ON BLOOD GLUCOSE METERS† • Kills C DIFF Spores in ONE STEP in 3 minutes • Kills +HBV & +HCV in 30 seconds

microdot®

BLEACH WIPE

SPORICIDAL • BACTERICIDAL • FUNGICIDAL • +VIRUCIDAL

Ready to use EPA registered bleach wipe

ONE STEP CLEANER AND DISINFECTANT THAT KILLS

Clostridium difficile spores, MRSA, VRE, Staphylococcus aureus, Salmonella enterica, Pseudomonas aeruginosa, Trichophyton mentagrophytes, +HIV-1 and +HBV. Preclean prior to disinfecting surfaces for +HCV.

Disinfects • Cleans • Deodorizes • Kills 99.99% of Germs

Meets OSHA blood borne pathogen standards in 29 CFR 1910.1030 and guidelines of the CDC and CMS Tag F441. Aids in reducing cross contamination between treated surfaces. A one step cleaning and disinfecting sporicidal bleach wipe that meets the CDC, CMS Tag F441, SHEA-IDS, APIC and Joint Commission guidelines for killing Clostridium difficile spores.

CAN BE USED ON BLOOD GLUCOSE METERS.†

Disinfects hard, non-porous healthcare surfaces, patient care equipment and point-of-care equipment.

Highly Absorbent & Tear Resistant

Active Ingredient: Sodium hypochlorite .....	0.65%
Other Ingredients*.....	99.35%
Total.....	100.00%

\*Other ingredients do not include the weight of wipe.

KEEP OUT OF REACH OF CHILDREN

CAUTION

(See side panels for directions for use, and additional precautionary statements.)

EPA Reg. No. 69687-1-88459  
EPA Est. No. 69687-IL-001

Made in the U.S.A.

NET CONTENTS:

1 lb. 3 oz.  
160 Pre-saturated 6" x 6"  
(15.2cm x 15.2cm)  
WIPES

DISINFECTING WIPE FOR HARD, NON-POROUS HEALTHCARE SURFACES AND ENVIRONMENTAL SURFACES.

SUGGESTED AREAS OF USAGE: Blood Glucose Meters†, Medical, Dental and Laboratory Counters, Exam Tables, Carts, Patient Care Equipment, Point-of-Care Equipment, Digital Screens, on Plastics, Glazed Porcelain, Stainless Steel, Formica, Vinyl and Rubber Surfaces.

FIRST AID: If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice. Have the product container with you when calling a poison control center or doctor, or going for treatment.

DIRECTIONS FOR USE: It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

Disinfection:

- Always use personal protective equipment. †
- Open Microdot® Bleach Wipe canister.
- Remove pre-saturated 6"x 6" wipe.
- Apply towelette and wipe desired surface to be disinfected.
- A 30 second contact time is required to kill HBV and HCV. A 3 minute contact time is required to kill Clostridium difficile spores. A 5 minute contact time is required to kill HIV and other organisms listed on the label. Reapply as necessary to ensure that the surface remains wet for entire contact time.
- Allow surface to air dry and discard used wipe and empty canister (see storage and disposal).

TO OPEN CANISTER:

- Always use personal protective equipment. †
- Remove lid from canister.
- Remove seal if present and discard
- Lift flap in canister lid.
- Pull up corner of center wipe and snap off, twist to a point and thread through dispenser hole in lid.
- DO NOT PUSH FINGER THROUGH OPENING.
- Replace lid. Pull out exposed wipe and snap off. The next wipe pops up automatically.
- Snap center flap down when finished to retain moisture.
- Do not flush wipe into toilet.

+KILLS HIV-1, HBV AND HCV ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids and in which the surfaces/objects likely to be soiled with blood/body fluids can be associated with the potential for transmission of HIV-1 (associated with AIDS), HBV and HCV.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV AND HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

- PERSONAL PROTECTION: When handling items soiled with blood or body fluids, use disposable gloves, gown, masks and eye protection.
- CLEANING PROCEDURES: Blood/body fluids must be thoroughly cleaned from surfaces/objects before application of Microdot® Bleach Wipe.
- CONTACT TIME: Allow surface to remain wet for 30 seconds to kill HBV and HCV, for 3 minute to kill Clostridium difficile spores, and 5 minutes to kill HIV and other organisms listed on the label.
- DISPOSAL OF INFECTIOUS MATERIAL: Blood/body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.

Precautionary Statements:

Hazards to humans and domestic animals.

CAUTION: Causes moderate eye irritation. Avoid contact with eyes and clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet. Remove and wash contaminated clothing before reuse.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barriers or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

†This product is for the exterior surfaces of Blood Glucose Meters and is not for use on the needle and/or the monitors.

STORAGE AND DISPOSAL

Do not contaminate water, food or feed by storage or disposal.

Pesticide Storage: Keep these wipes in a tightly closed canister, when not in use. Store canister in cool, dry, and well-ventilated area away from heat, open flame, direct sunlight and inaccessible to children.

Pesticide (wipe) Disposal: Wastes resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.

Container Handling: Nonrefillable rigid canister. Do not re-use or refill this container. Triple rinse (or equivalent), promptly after emptying. Offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration, or, if allowed by state and local authorities, by burning. If burned, stay out of smoke.

Complete product line information available by calling:

Cambridge Sensors USA, LLC.  
10051 Bode Road  
Plainfield, Illinois 60585  
Customer Service: Toll Free 877.374.4062 (M-F 9am-5pm CST)  
info@microdotcs.com www.microdotcs.com

092613

CS®

CAMBRIDGE SENSORS USA, LLC

©2013 Cambridge Sensors USA, LLC.

4.500"