

CAUTION
Keep out of Reach of Children
See back panel for additional
precautionary statements.

Net contents: 50 Tablets • 0.26 oz. total

Inhibits the Growth of Microbial Contamination in Dental Unit Waterlines

Dental Unit Waterline Treatment

Z³ Waterline Maintenance Tablets

.7L Tablets

Benco Dental™ Z³™ Waterline Maintenance Tablets

Clinical monitoring of water quality can ensure that procedures are being performed correctly and are effective. Test water quality monthly to establish effectiveness of your maintenance program. If water quality results approach or exceed requirements, repeat Initial Treatment.

PRECAUTIONARY STATEMENTS: Hazards to Humans and Domestic Animals.

CAUTION: Tablet may cause moderate eye irritation. Avoid contacting tablet with eyes or clothing. Wash thoroughly with soap and water after handling tablets and before eating, drinking, chewing gum, or using tobacco.

FIRST AID: If in Eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first five minutes and continue rinsing. Call a poison control center or doctor for treatment advice.

HOTLINE NUMBER: In case of emergency, call toll free 1-866-290-1830. Have the product container or label with you when calling a poison control center or doctor or going for treatment.

Benco Dental Z³ Waterline Maintenance Tablets are designed to be continuously present in your dental unit water system.

STATE OF HAWAII
Department of Agriculture
ACCEPTED
LICENSE NO.
8924.1

Benco Dental™ Z³™ Waterline Maintenance Tablets

STORAGE AND DISPOSAL
STORAGE: Do not contaminate water, food, or feed by storage or disposal. Store in a cool, dry place.

FOIL POUCH
DISPOSAL: If empty, place in trash or offer for recycling if available. If partly filled, call your solid waste agency (or 1-800-CLEANUP) for disposal instructions.

OUTER CONTAINER
DISPOSAL: Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available or dispose of the empty container in trash.

.7L Tablets - Reorder # [4600-362]



Distributed by Benco Dental Supply Company

Benco Dental
We deliver success smile after smile.
866.GO.BENCO - benco.com
295 Conestoga Blvd., Pottsville, PA 19864

EPA Reg. # 84020-1-35659
EPA Establishment # 84065-MO-001

Made in USA

Benco Dental™ Z³™ Waterline Maintenance Tablets

Dental Unit Waterline Treatment
Inhibits the Growth of Microbial Contamination in Dental Unit Waterlines

.7L Tablets

Net contents: 50 Tablets • 0.26 oz. total

ACTIVE INGREDIENT:

Silver	0.477%
Other Ingredients	99.523%
TOTAL	100.000%

CAUTION
Keep out of Reach of Children
See back panel for additional precautionary statements.

Benco Dental™ Z³™ Waterline Maintenance Tablets

DIRECTIONS FOR USE: IT IS A VIOLATION OF FEDERAL LAW TO USE THIS PRODUCT IN A MANNER INCONSISTENT WITH ITS LABELING. READ ENTIRE LABEL BEFORE USING THIS PRODUCT.

INITIAL TREATMENT: Before starting a treatment program with Benco Dental Z³ Waterline Maintenance Tablets, flush lines with a waterline cleaner or another product recommended by the equipment manufacturer to remove line buildup or deposits. When flushing dental unit water lines, always follow a recognized line flushing/cleaning protocol. Repeat flushing to clean lines every 2 months or more frequently when water quality results indicate.

ROUTINE TREATMENT:

- Place tablet or tablets (see table below) in empty dental unit water bottle every time you refill the water. Benco Dental Z³ Waterline Maintenance Tablets are intended for use with potable water (including tap or distilled).
- Fill bottle with desired volume of water and connect to dental unit. Tablet dissolves in a few minutes.
- Benco Dental Z³ Waterline Maintenance Tablets may be left in the waterlines overnight and for extended periods of non-use. Use water solution until it is expended or up to 28 days. It is not necessary to purge lines at the end of each day.

Water Volume	Tablets
700-750ml (24-25 fl. oz.)	1
1500ml (51 fl. oz.)	2
2000ml (68 fl. oz.)	3