




STATE OF HAWAII
Department of Agriculture

ACCEPTED

LICENSE NO.

9203.311

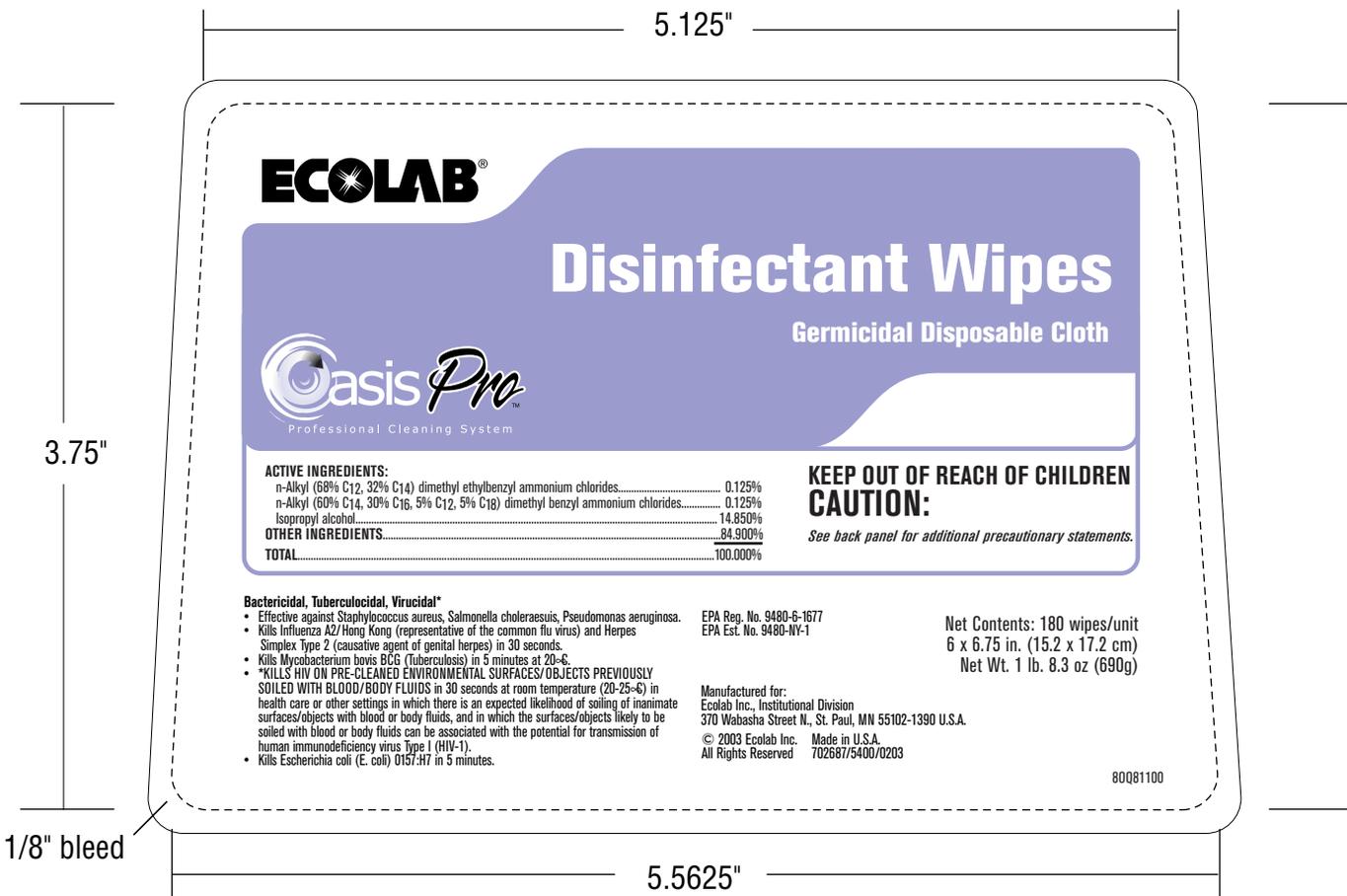

ECOLAB®

Disinfectant Wipes

Germicidal Disposable Cloth



ACTIVE INGREDIENTS:

n-Alkyl (68% C12, 32% C14) dimethyl ethylbenzyl ammonium chlorides.....	0.125%
n-Alkyl (60% C14, 30% C16, 5% C12, 5% C18) dimethyl benzyl ammonium chlorides.....	0.125%
Isopropyl alcohol.....	14.850%
OTHER INGREDIENTS.....	84.900%
TOTAL.....	100.000%

**KEEP OUT OF REACH OF CHILDREN
CAUTION:**

See back panel for additional precautionary statements.

Bactericidal, Tuberculocidal, Virucidal*

- Effective against *Staphylococcus aureus*, *Salmonella choleraesuis*, *Pseudomonas aeruginosa*.
- Kills Influenza A2/Hong Kong (representative of the common flu virus) and Herpes Simplex Type 2 (causative agent of genital herpes) in 30 seconds.
- Kills *Mycobacterium bovis* BCG (Tuberculosis) in 5 minutes at 20-6°C.
- ***KILLS HIV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in 30 seconds at room temperature (20-25-6) in health care or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1).
- Kills *Escherichia coli* (E. coli) O157:H7 in 5 minutes.

EPA Reg. No. 9480-6-1677
EPA Est. No. 9480-NY-1

Net Contents: 180 wipes/unit
6 x 6.75 in. (15.2 x 17.2 cm)
Net Wt. 1 lb. 8.3 oz (690g)


Manufactured for:
Ecolab Inc., Institutional Division
370 Wabasha Street N., St. Paul, MN 55102-1390 U.S.A.

© 2003 Ecolab Inc. Made in U.S.A.
All Rights Reserved 702687/5400/0203

80Q81100

702687/54 s inks: PMS 2645 Light Purple & Black Remove dashed border before printing

160/140ct Canister Front

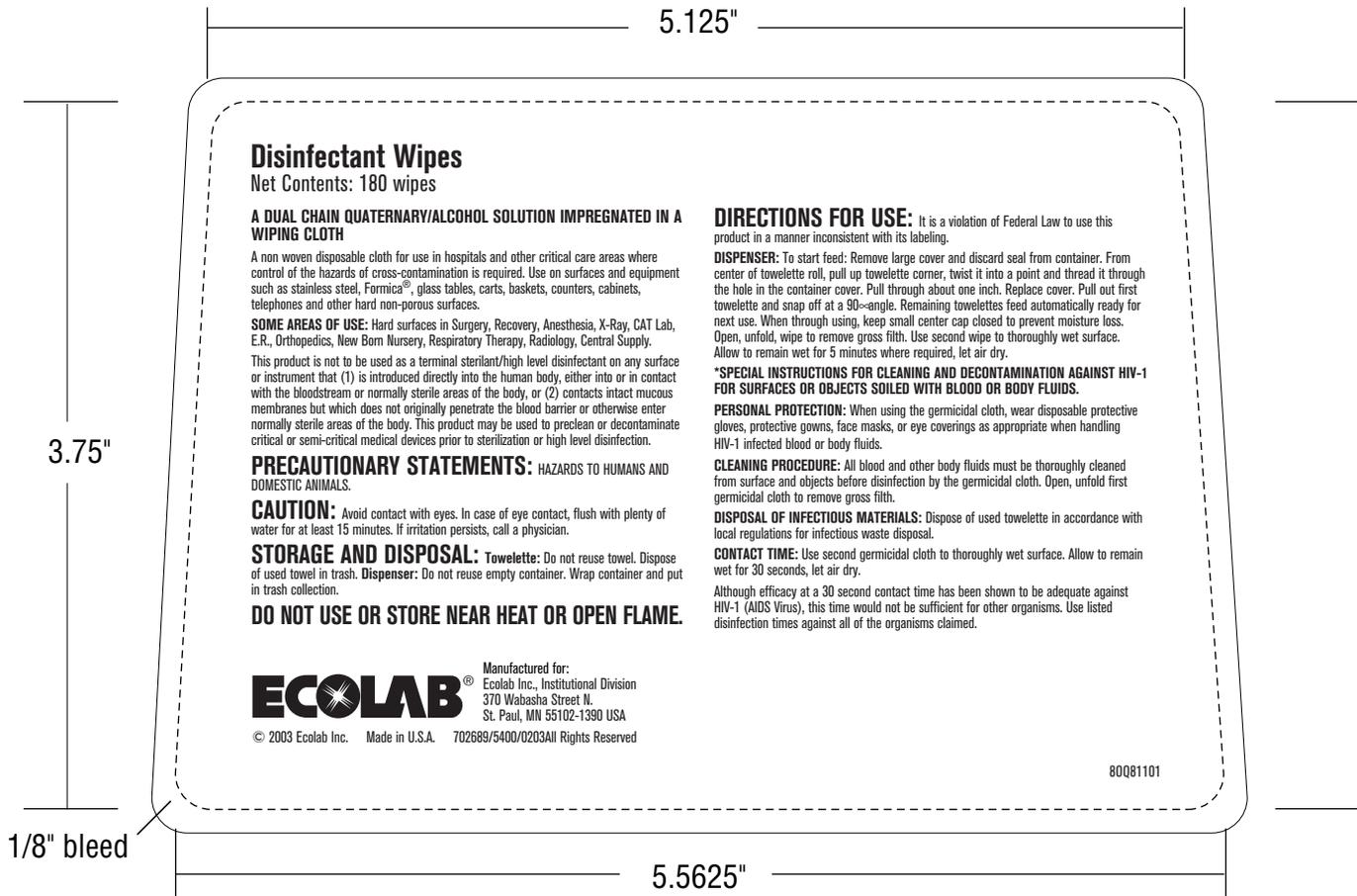

Disinfectant Wipes

Net Contents: 180 wipes

A DUAL CHAIN QUATERNARY/ALCOHOL SOLUTION IMPREGNATED IN A WIPING CLOTH

A non woven disposable cloth for use in hospitals and other critical care areas where control of the hazards of cross-contamination is required. Use on surfaces and equipment such as stainless steel, Formica®, glass tables, carts, baskets, counters, cabinets, telephones and other hard non-porous surfaces.

SOME AREAS OF USE: Hard surfaces in Surgery, Recovery, Anesthesia, X-Ray, CAT Lab, E.R., Orthopedics, New Born Nursery, Respiratory Therapy, Radiology, Central Supply.

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not originally penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

PRECAUTIONARY STATEMENTS: HAZARDS TO HUMANS AND DOMESTIC ANIMALS.

CAUTION: Avoid contact with eyes. In case of eye contact, flush with plenty of water for at least 15 minutes. If irritation persists, call a physician.

STORAGE AND DISPOSAL: **Towelette:** Do not reuse towel. Dispose of used towel in trash. **Dispenser:** Do not reuse empty container. Wrap container and put in trash collection.

DO NOT USE OR STORE NEAR HEAT OR OPEN FLAME.



Manufactured for:
Ecolab Inc., Institutional Division
370 Wabasha Street N.
St. Paul, MN 55102-1390 USA

© 2003 Ecolab Inc. Made in U.S.A. 702689/5400/0203AH Rights Reserved

DIRECTIONS FOR USE: It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

DISPENSER: To start feed: Remove large cover and discard seal from container. From center of towelette roll, pull up towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90°-angle. Remaining towelettes feed automatically ready for next use. When through using, keep small center cap closed to prevent moisture loss. Open, unfold, wipe to remove gross filth. Use second wipe to thoroughly wet surface. Allow to remain wet for 5 minutes where required, let air dry.

***SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS.**

PERSONAL PROTECTION: When using the germicidal cloth, wear disposable protective gloves, protective gowns, face masks, or eye coverings as appropriate when handling HIV-1 infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surface and objects before disinfection by the germicidal cloth. Open, unfold first germicidal cloth to remove gross filth.

DISPOSAL OF INFECTIOUS MATERIALS: Dispose of used towelette in accordance with local regulations for infectious waste disposal.

CONTACT TIME: Use second germicidal cloth to thoroughly wet surface. Allow to remain wet for 30 seconds, let air dry.

Although efficacy at a 30 second contact time has been shown to be adequate against HIV-1 (AIDS Virus), this time would not be sufficient for other organisms. Use listed disinfection times against all of the organisms claimed.

80Q81101

702689/54 inks: Black Remove dashed border before printing.

160/140ct Canister Back