

DS CONTAINERS 211x713 Decoration Layout

MEDAPHENE® Plus

MEDAPHENE® Plus Disinfectant Spray is a general purpose disinfectant/deodorant that is Bactericidal, Tuberculocidal, Fungicidal, and Virucidal*. The broad spectrum germicidal efficacy of MEDAPHENE® Plus Disinfectant Spray has been established, with a ten (10) minute contact time at 25 degrees Celsius, against the following organisms:

BACTERICIDAL: Mycobacterium tuberculosis, Staphylococcus aureus, Salmonella choleraesuis, Pseudomonas aeruginosa, and Methicillin-Resistant Staphylococcus aureus (MRSA). ***VIRUCIDAL:** HIV-1 (AIDS Virus), Herpes Simplex Virus (Type 2), Influenza A (Hong Kong), Avian influenza virus, Avian bronchitis virus, Avian laryngotracheitis virus, Hog cholera virus, and Pseudorabies virus on hard, nonporous, inanimate surfaces. Kills Pandemic 2009 H1N1 Influenza A virus (formerly called swine flu). **FUNGICIDAL:** Trichophyton mentagrophytes pathogenic fungi.

Medaphene® Plus Disinfectant Spray can be used to clean and disinfect floors, benches, and other hard, nonporous surfaces commonly found in bathrooms, shower rooms, and locked rooms.

GENERAL USES

MEDAPHENE® Plus Disinfectant Spray is an effective disinfectant on hard, nonporous, inanimate surfaces and objects such as bed frames, desks, tables, chairs, file cabinets, cabinets, countertops, door handles, walls, sinks, floors, light switch covers, linen carts, telephones, waste containers, hampers, exterior bowl surfaces and exterior urinal surfaces. **DO NOT USE ON POLISHED WOOD FURNITURE OR RAYON FABRICS.** NOTE: This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling. Shake well before using.

TO DEODORIZE: Hold can 6-8 inches from surface. Spray surfaces. Then wipe.

As a MILDEW PREVENTATIVE: Spray surface until moist. Do not use on polished wood furniture or rayon fabrics. **As a DISINFECTANT:** Thoroughly clean all surfaces with soap and detergent and rinse with water. Hold can upright 6 to 8 inches from surface. Saturate all surfaces with disinfecting solution, allowing to remain wet for 10 minutes before wiping, if wiping is required.

***KILLS HIV-1 ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in health care setting or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of human immunodeficiency virus Type 1 (HIV-1) (associated with AIDS).

SPECIAL INSTRUCTION FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

PERSONAL PROTECTION: When handling items soiled with blood or body fluids, use disposable latex gloves, gowns, masks, and eye coverings.

CLEANING PROCEDURE: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant.

CONTACT TIME: Allow surface to remain wet for 10 minutes.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids must be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals

WARNING

Harmful if absorbed through skin. Causes moderate eye irritation. Avoid contact with eyes, skin or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. If on clothing, remove contaminated clothing and wash before reuse.

PHYSICAL OR CHEMICAL HAZARDS

Flammable. Contents under pressure. Keep away from heat, sparks and open flame. Do not smoke while using product. Do not puncture or incinerate container. Exposure to temperatures above 130°F may cause bursting.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage or disposal. Store in a cool, dry place away from heat or open flame. Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available. If not, discard in trash.

Notice: California has determined that a chemical contained in this product causes cancer based on tests performed on laboratory animals.

For Commercial/Institutional Use Only

EPA Reg. No. 211-32-11694 SDA-KS-774
EPA Est. No. (1)11623-GA-1, (2)11623-GA-2, (3)11694-KS-1

Manufactured in the U.S.A. by

ITW PRO BRANDS

805 E. Old 56 Highway • Olathe, KS 66061
itwprobrands.com • 800.443.9536
Emergency 800.535.5053 Infotrac

 You can recycle this steel container in an increasing number of communities.

online SDS
www.dymon.com
PRODUCT # 34720



MEDAPHENE® Plus

Disinfectant Spray

ACTIVE INGREDIENTS:

o-phenylphenol	0.210%
ethyl alcohol	69.623%
INERT INGREDIENTS:	30.167%
TOTAL	100.000%

KEEP OUT OF REACH OF CHILDREN

WARNING

See Additional Precautionary Statements on Back Panel

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a Poison Control Center or doctor for treatment advice. **If on skin or clothing:** Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a Poison Control Center or doctor for further treatment advice. In case of an emergency, call your local Poison Control Center or a doctor. **Have the product container or label with you when calling a Poison Control Center or doctor or going for treatment.**

NET WT 16 OZ (1 lb) (454 g)



STATE OF HAWAII
Department of Agriculture

ACCEPTED

LICENSE NO.
9576.45

7 64769 34720 1
140819



1/23/13

ITW Dymon

347
MEDAPHENE Plus 20 oz

- PMS 293 Blue
- PMS 186 Red
- PMS 3265 C
- Black

Art prepared by:
Leinicke Group
213 Old Meramec Station Rd
Manchester, MO 63021
636-227-4424