

MAXISPRAY PLUS EFFECTIVELY KILLS HUMAN IMMUNODEFICIENCY VIRUS TYPE 1 (HIV), HEPATITIS B VIRUS (HBV), AND HEPATITIS C VIRUS (HCV) ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS IN HEALTHCARE SETTINGS OR OTHER SETTINGS IN WHICH THERE IS AN EXPECTED LIKELIHOOD OF SOILING OF INANIMATE SURFACES/OBJECTS WITH BLOOD/BODY FLUIDS, AND IN WHICH THE SURFACES/OBJECTS CAN BE ASSOCIATED WITH THE POTENTIAL FOR TRANSMISSION OF HIV-1, HBV, AND HCV.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV, AND HCV OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUID:

Personal Protection: Wear appropriate barrier protection such as protective gloves, gowns, masks or eye coverings.

Cleaning Procedure: Blood and other bodily fluids must be thoroughly cleaned from surfaces and objects before disinfection with MaxiSpray Plus.

Contact Time: A contact time of 2 minutes is required for HIV-1, HBV and HCV. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.

Infectious Materials Disposal: Cleaning materials used that may contain blood or other bodily fluids should be autoclaved and/or disposed of in accordance with local regulations for infectious materials disposal.

This product is not to be used as a terminal sterilant/ high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

Pesticide Storage: Store in a cool place.

Pesticide Disposal: Wastes resulting from the use of this product may be disposed of according to applicable Federal, State, or local procedures.

Container Disposal: Triple rinse. Then offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration or, if allowed by state and local authorities, by burning. If burned, keep out of smoke.

Distributed By:

HENRY SCHEIN INC.
Melville, NY 11747 USA

MaxiSpray Plus® is a registered trademark of Henry Schein, Inc.

MADE IN USA

Production lot # is stamped on the container.

For product information, please contact our technical service department at 1-800-472-4346.

70-7400-8



+H658101028508



Department of Agriculture
STATE OF HAWAII

LICENSED

PERIOD **2011-2013** LIC. NO.



Department of Agriculture
STATE OF HAWAII

LICENSED

PERIOD **2008-2010** LIC. NO.



HENRY SCHEIN®

MaxiSpray®

**SURFACE DISINFECTANT
DECONTAMINANT CLEANER**

- Bactericidal
- Virucidal*
- Tuberculocidal**
- Fungicidal

READY-TO-USE
FOR PROFESSIONAL USE

ACTIVE INGREDIENTS:


Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride	0.28%
Isopropanol	17.20%
INERT INGREDIENTS:	82.52%
TOTAL	100.00%

KEEP OUT OF REACH OF CHILDREN
CAUTION

NET CONTENTS: ONE GALLON (128 FL. OZ.)(3.8 L)
(Net Weight 8.22 lbs. / 3.73 kg)

REFILL

101-0285

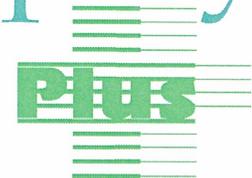

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals.
CAUTION

Harmful if absorbed through skin. Causes moderate eye irritation. Avoid contact with eyes, skin or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse. See other panel for additional precautionary statements.

Physical or Chemical Hazards: Combustible. Do not use or store near heat or open flame.

EPA Reg. No. 46781-6-10597
EPA Est. No. 46781-MI-1



Department of Agriculture
STATE OF HAWAII

LICENSED

PERIOD **2014-2016** LIC. NO.

9591.43

101-0285



ic Animals.

auses moderate
s, skin or clothing.
), chewing gum,
remove
othing before reuse.
autionary

nbustible. Do not use

L)

EFFECTIVE AGAINST

*Mycobacterium tuberculosis var: bovis (BCG)***
Staphylococcus aureus
Pseudomonas aeruginosa
Salmonella choleraesuis
Trichophyton mentagrophytes
Methicillin Resistant *Staphylococcus aureus* (MRSA)
Vancomycin Resistant *Enterococcus faecalis* (VRE)
Hepatitis B Virus (HBV)*
Hepatitis C Virus (HCV)*
Herpes simplex virus Types 1 and 2*
Human Immunodeficiency Virus (HIV-1)*

DESCRIPTION

MaxiSpray Plus is a multi-purpose disinfectant/decontaminant cleaner for use on hard, non-porous inanimate surfaces.

It is useful in hospital operating rooms, neonatal units, dental operatories, surgical suites, animal care facilities, out-patient surgical centers, and other critical care areas where control of cross contamination between treated surfaces is important.

MaxiSpray Plus will effectively clean and disinfect, when used as directed, such items as interior and exterior surfaces of infant incubators and bassinets, exterior surfaces of anesthesia machines and respiratory therapy equipment surfaces, operating room tables and lights, physical therapy (PT) equipment surfaces, animal cages, veterinary care surfaces, exterior surfaces of dental operatories, dental countertops, dental chairs, stools, exterior surfaces of dental curing lights, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

FOR PRECLEANING NON-INSTRUMENT SURFACES PRIOR TO DISINFECTION:

Remove gross filth and heavy soil loads. Spray MaxiSpray Plus directly onto surface. Allow to remain visibly wet for 30 seconds. Wipe surface using a towel.

FOR PRECLEANING INSTRUMENTS PRIOR TO DISINFECTION:

Remove gross filth and heavy soil loads.

For use as a precleaning decontaminant immersion solution: Place instruments in a container with a sufficient amount of MaxiSpray Plus so as to allow for complete submersion, cover and immerse for 10 minutes at room temperature (69 °F/20 °C). Remove and rinse instruments. Follow with appropriate cleaning and disinfection process. Discard solution after each use.

For use as a precleaning decontaminant spray: Place instruments in a container. Spray MaxiSpray Plus so as to thoroughly drench all instrument surfaces and allow to

remain visibly wet for 30 seconds. Remove and rinse instruments. Follow with appropriate cleaning and disinfection process.

For use as an ultrasonic precleaning solution: Thoroughly rinse instruments under running water to remove visible gross debris. Add 1 ounce MaxiSpray Plus per liter of water to ultrasonic unit. Immerse instruments in solution and activate ultrasonic unit for 5 minutes or longer, if necessary. Remove and rinse instruments. Follow with appropriate cleaning and disinfection process. Discard solution after each use.

For use as a manual precleaner: Thoroughly rinse instruments under running water to remove visible gross debris. Place instruments in a container with 1 ounce MaxiSpray Plus per liter of water. Scrub instruments while submerged for 30 seconds using a stiff bristle brush until visibly clean. Remove and rinse instruments. Follow with appropriate cleaning and disinfection process.

FOR USE AS A VIRUCIDE* ON NON-INSTRUMENT SURFACES, INCLUDING HIV-1, HBV AND HCV APPLICATIONS:

Spray MaxiSpray Plus directly onto precleaned surface, thoroughly wetting area to be treated. Allow surface to remain visibly wet for 2 minutes at room temperature (69 °F/20 °C)

FOR USE AS A DISINFECTANT ON NON-INSTRUMENT SURFACES:

Spray MaxiSpray Plus directly onto precleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain visibly wet for 3 minutes at room temperature (69 °F/20 °C) for Tuberculocidal** activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella choleraesuis*. Wipe surface using a towel or allow to air dry.

FOR USE AS A VIRUCIDE* ON PRECLEANED NON-CRITICAL MEDICAL DEVICES; INSTRUMENTS AND IMPLEMENTS INCLUDING HIV-1, HBV AND HCV APPLICATIONS:

Instruments must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow instruments. Using either a tray or an ultrasonic unit, immerse instruments in MaxiSpray Plus for 2 minutes at room temperature (69 °F/20 °C).

FOR USE AS A DISINFECTANT ON PRECLEANED NON-CRITICAL MEDICAL DEVICES; INSTRUMENTS AND IMPLEMENTS:

Instruments must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow instruments. Using either a tray or an ultrasonic unit, immerse instruments in MaxiSpray Plus for 3 minutes at room temperature (69 °F/20 °C) for Tuberculocidal** activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella choleraesuis*.

Remove and rinse instruments. Wipe dry prior to use. Discard solution after each use.

* Non-critical medical devices are items that come in contact only with intact skin.

Note: Critical and semi-critical devices must be followed by appropriate terminal sterilization/high-level disinfection process.



MAXISPAY PLUS EFFECTIVELY KILLS HUMAN IMMUNODEFICIENCY VIRUS TYPE 1 (HIV), HEPATITIS B VIRUS (HBV), AND HEPATITIS C VIRUS (HCV) ON PRECLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS IN HEALTHCARE SETTINGS OR OTHER SETTINGS IN WHICH THERE IS AN EXPECTED LIKELIHOOD OF SOILING OF INANIMATE SURFACES/OBJECTS WITH BLOOD/BODY FLUIDS, AND IN WHICH THE SURFACES/OBJECTS CAN BE ASSOCIATED WITH THE POTENTIAL FOR TRANSMISSION OF HIV-1, HBV, AND HCV.

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV, AND HCV OF SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUID:

Personal Protection: Wear appropriate barrier protection such as protective gloves, gowns, masks or eye coverings.

Cleaning Procedure: Blood and other bodily fluids must be thoroughly cleaned from surfaces and objects before disinfection with MaxiSpray Plus.

Contact Time: A contact time of 2 minutes is required for HIV-1, HBV and HCV. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.

Infectious Materials Disposal: Cleaning materials used that may contain blood or other bodily fluids should be autoclaved and/or disposed of in accordance with local regulations for infectious materials disposal. This product is not to be used as a terminal sterilant/ high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either

into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

FIRST AID

If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye. Call a poison control center or doctor for treatment advice.

If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice. For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

Pesticide Storage: Store in a cool place.

Pesticide Disposal: Wastes resulting from the use of this product may be disposed of according to applicable Federal, State, or local procedures.

Container Disposal: Triple rinse. Then offer for recycling or reconditioning, or puncture and dispose of in a sanitary landfill, or incineration or, if allowed by state and local authorities, by burning. If burned, keep out of smoke.

Distributed By: 70-7405-8
HENRY SCHEIN INC.
 Melville, NY 11747 USA MADE IN USA
 MaxiSpray Plus® is a registered trademark of Henry Schein, Inc.
 For product information, please contact our technical service department at 1-800-472-4346.



+H658101411403



HENRY SCHEIN®

101-4114

MaxiSpray®

SURFACE DISINFECTANT
 DECONTAMINANT CLEANER

Pump Spray



- Bactericidal
- Virucidal*
- Tuberculocidal**
- Fungicidal

READY-TO-USE • FOR PROFESSIONAL USE

ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride.....	0.28%
Isopropanol	17.20%
INERT INGREDIENTS:	82.52%
TOTAL:	100.00%

KEEP OUT OF REACH OF CHILDREN

CAUTION

PRECAUTIONARY STATEMENTS

Hazards to Humans and Domestic Animals.

CAUTION

Harmful if absorbed through skin. Causes moderate eye irritation. Avoid contact with eyes, skin or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse. See other panel for additional precautionary statements.

Physical or Chemical Hazards: Combustible. Do not use or store near heat or open flame.

EPA Reg. No. 46781-6-10597 EPA Est. No. 46781-MI-1

NET CONTENTS: 24 FL. OZ. (709 ML)
 (Net Weight 1.537 lbs. / 700 g)

EFFECTIVE AGAINST

*Mycobacterium tuberculosis var: bovis (BCG)***

Staphylococcus aureus

Pseudomonas aeruginosa

Salmonella choleraesuis

Trichophyton mentagrophytes

Methicillin Resistant *Staphylococcus aureus* (MRSA)

Vancomycin Resistant *Enterococcus faecalis* (VRE)

Hepatitis B Virus (HBV)*

Hepatitis C Virus (HCV)*

Herpes simplex virus Types 1 and 2*

Human Immunodeficiency Virus (HIV-1)*

DESCRIPTION

MaxiSpray Plus is a multi-purpose disinfectant/decontaminant cleaner for use on hard, non-porous inanimate surfaces.

It is useful in hospital operating rooms, neonatal units, dental operatories, surgical suites, animal care facilities, out-patient surgical centers, and other critical care areas where control of cross contamination between treated surfaces is important. MaxiSpray Plus will effectively clean and disinfect, when used as directed, such items as interior and exterior surfaces of infant incubators and bassinets, exterior surfaces of anesthesia machines and respiratory therapy equipment surfaces, operating room tables and lights, physical therapy (PT) equipment surfaces, animal cages, veterinary care surfaces, exterior surfaces of dental operatories, dental countertops, dental chairs, stools, exterior surfaces of dental curing lights, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

FOR PRECLEANING NON-INSTRUMENT SURFACES PRIOR TO DISINFECTION:

Remove gross filth and heavy soil loads. Spray MaxiSpray Plus directly onto surface. Allow to remain visibly wet for 30 seconds. Wipe surface using a towel.

FOR PRECLEANING INSTRUMENTS PRIOR TO DISINFECTION:

Remove gross filth and heavy soil loads.

For use as a precleaning decontaminant spray: Place instruments in a container. Spray MaxiSpray Plus so as to

thoroughly drench all instrument surfaces and allow to remain visibly wet for 30 seconds. Remove and rinse instruments. Follow with appropriate cleaning and disinfection process.

FOR USE AS A VIRUCIDE* ON NON-INSTRUMENT SURFACES, INCLUDING HIV-1, HBV AND HCV APPLICATIONS:

Spray MaxiSpray Plus directly onto precleaned surface, thoroughly wetting area to be treated. Allow surface to remain visibly wet for 2 minutes at room temperature (69 °F/20 °C)

FOR USE AS A DISINFECTANT ON NON-INSTRUMENT SURFACES:

Spray MaxiSpray Plus directly onto precleaned surface, thoroughly wetting area to be disinfected. Allow surface to remain visibly wet for 3 minutes at room temperature (69 °F/20 °C) for Tuberculocidal** activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella choleraesuis*. Wipe surface using a towel or allow to air dry.

FOR USE AS A VIRUCIDE* ON PRECLEANED NON-CRITICAL MEDICAL DEVICES, INSTRUMENTS AND IMPLEMENTS INCLUDING HIV-1, HBV AND HCV APPLICATIONS:

Instruments must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow instruments. Using either a tray or an ultrasonic unit, immerse instruments in MaxiSpray Plus for 2 minutes at room temperature (69 °F/20 °C).

FOR USE AS A DISINFECTANT ON PRECLEANED NON-CRITICAL MEDICAL DEVICES, INSTRUMENTS AND IMPLEMENTS:

Instruments must be thoroughly cleaned to remove excess organic debris, rinsed and dried. Thoroughly clean and rinse lumens of hollow instruments. Using either a tray or an ultrasonic unit, immerse instruments in MaxiSpray Plus for 3 minutes at room temperature (69 °F/20 °C) for Tuberculocidal** activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella choleraesuis*.

Remove and rinse instruments. Wipe dry prior to use. Discard solution after each use.

* Non-critical medical devices are items that come in contact only with intact skin.

Note: Critical and semi-critical devices must be followed by appropriate terminal sterilization/high-level disinfection process.

