

DO NOT REMOVE THIS INFORMATION!
IT IS IMPORTANT FOR PRODUCTION WORKFLOW. ROCKLINE

DIE LINE #	2732G	LABEL, 16 11/16" x 4 1/4" FOR 120 MM CANISTER	STACK (mm) N/A
SIZE	16-11/16 x 4-1/4 (+/- 1/32)		VIEW: PRINT SIDE
PRODUCT	120 MM CANISTER PRODUCT		KEY CUT: _____ NO PRINT OR COPY AREA: _____ NO COATING AREA: _____
OTHER INFO:	PRINT PRESS POSITION IN TOP RIGHT CORNER OF GLUE FLAP IN 3 PT. HELVETICA BOLD		



LICENSED

PERIOD 2014-2016 LIC. NO. 9591.73

16^{11/16}

Special Instructions for Cleaning and Decontamination Against HIV-1*, HBV* and HCV* of Surfaces or Objects Soiled With Blood or Body Fluids:
Personal Protection: When using this disinfecting towelette, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.
Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A MaxiWipe LA towelette can be used for this purpose.
Contact Time: Thoroughly wet surface with a new MaxiWipe LA towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.
Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids must be autoclaved and disposed of in accordance with federal, state and local regulations for infectious materials disposal.

CAUTION
Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse.

FIRST AID
If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for further advice.
If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300. For ALL other inquiries about this product, call Henry Schein at 1-800-472-4346, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL
Towelette - Do not reuse towelette. Dispose of used towelette in trash.
Dispenser - Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available.

Distributed by:
Henry Schein, Inc.
135 Duryea Rd.
Melville, NY 11747

MADE IN USA

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals

HENRY SCHEIN®

MAXIWipe® LA

Tuberculocidal • Bactericidal • Virucidal • Fungicidal

DISINFECTING TOWELETTES

112-6844

MaxiWipe LA Disinfecting Towelettes are non-woven disposable towelettes intended for use in healthcare settings such as hospitals, dental offices, laboratories, clinics, ophthalmic offices, veterinary facilities and in kitchens** and bathrooms. MaxiWipe LA is effective against the following microorganisms on hard, non-porous surfaces when used as directed:

- *Mycobacterium tuberculosis* var: *bovis* (BCG) ■ *Staphylococcus aureus* ■ *Pseudomonas aeruginosa*
- *Salmonella enterica* ■ *Trichophyton mentagrophytes* ■ Methicillin Resistant *Staphylococcus aureus* (MRSA)
- Vancomycin Resistant *Enterococcus faecalis* (VRE) ■ *Staphylococcus aureus* with Reduced Susceptibility to Vancomycin ■ Hepatitis B Virus (HBV)* ■ Hepatitis C Virus (HCV)* ■ Herpes Simplex Virus Types 1 and 2*
- Human Immunodeficiency Virus (HIV-1)* ■ Influenza A2 Virus*

ACTIVE INGREDIENTS:	
Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride	0.28%
Isopropanol	17.20%
INERT INGREDIENTS:	82.52%
TOTAL	100.00% ¹

¹Does not include the weight of the dry towelette.

Net Contents:

65 Pre-Saturated Towelettes (9" x 12")

Net Weight: 2 lb. 1 oz.

See other panel for additional
precautionary statements.

EPA Reg. No. 46781-8-10597
EPA Est. No. 56952-WI-001



KEEP OUT OF REACH
OF CHILDREN
CAUTION

DO NOT USE AS A BABY WIPE. Not For Use On Skin.
DIRECTIONS FOR USE

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

MaxiWipe LA Disinfecting Towelettes are non-woven disposable towelettes intended for use on hard, non-porous surfaces in healthcare settings such as hospitals, emergency medical settings, neonatal units, surgical centers, isolation areas, out-patient surgical centers, nursing homes, dental offices, ophthalmic offices, eye surgical centers, patient care areas, dialysis centers, donation centers (blood, plasma, semen, milk, aphaeresis), veterinary facilities, salon settings, emergency vehicles, correctional facilities, schools, health club facilities, tanning salons, kitchens**, non-food contact surfaces in food preparation areas**, bathrooms, day care centers, police and fire vehicles, cadaver processing areas, laundry rooms, and other critical care areas where control of cross contamination between treated surfaces is required.

MaxiWipe LA will clean and disinfect, when used as directed, such items as interior and exterior surfaces of infant incubators, bassinets, cribs and warmers, infant/child-care equipment surfaces, diaper changing stations, oxygen hoods, operating room tables and lights, laboratory equipment and surfaces, physical therapy equipment surfaces, ambulance equipment surfaces, slit lamps, hard, non-porous exterior surfaces of ultrasound transducers or probes, exam tables, gurneys, IV poles, dental unit instrument trays, operator light switches, exterior surfaces of amalgamators and dental curing lights, exterior surfaces of anesthesia machines and respiratory therapy equipment, SilkPeel™ with Dermalfusion™, workstations, dental counter tops, loupes, exterior surfaces of endodontic equipment, such as apex locators, pulp testers and motors, stools, nail/hair care implements, tanning beds, salon surfaces, non-food contact surfaces**, exterior surfaces of refrigerator units**, exterior surfaces of appliances such as refrigerators, microwaves, food processors and stoves**, vanity tops**, floors, walls, handrails, door knobs, bed railings, tables, chairs, bathroom fixtures, exterior toilet surfaces, garbage cans, keyboards, carts, shopping carts, weight-lifting surfaces, drained whirlpool tank surfaces, cabinets, lights, telephones, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinyl chloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

Kitchen and Bath Wipes for Hard, Non-Porous Surfaces Cleans,**

Deodorizes and Kills Common Household Germs

**A thorough rinse with potable water is required for surfaces in direct contact with food.

Dispenser Directions

To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

Cleaning Instructions:

Use one MaxiWipe LA towelette to completely pre-clean surface of all gross debris.
For use as a virucide*, including HIV-1*, HBV* and HCV* applications and effectiveness Against Methicillin Resistant *Staphylococcus aureus* (MRSA), Vancomycin Resistant *Enterococcus faecalis* (VRE) and *Staphylococcus aureus* with Reduced Susceptibility to Vancomycin: Use a second MaxiWipe LA towelette to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes at room temperature (68°F/20°C).

For use as a disinfectant:

MaxiWipe LA is a two-step disinfectant product. One wipe is required to pre-clean then a second wipe is required to disinfect. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at room temperature (68°F/20°C) for Tuberculocidal activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella enterica*.

To disinfect external surfaces of ultrasound transducers:

Pre-clean ultrasound gel from external transducer to be disinfected. Then thoroughly wet the probe with a MaxiWipe LA towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at 68°F/20°C. **MaxiWipe LA** effectively kills Human Immunodeficiency Virus Type 1 (HIV-1)*, Hepatitis B Virus (HBV)* and Hepatitis C Virus (HCV)* on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of HIV-1*, HBV* and HCV*.

4^{1/4}

← LOT CODE AREA: NO COPY →

**DO NOT REMOVE THIS INFORMATION!
IT IS IMPORTANT FOR PRODUCTION WORKFLOW. ROCKLINE**

DIE LINE #	2732G	LABEL, 16 11/16" x 4 1/4" FOR 120 MM CANISTER	STACK (mm) N/A
SIZE	16-11/16 x 4-1/4 (+/- 1/32)		VIEW: PRINT SIDE
PRODUCT	120 MM CANISTER PRODUCT		KEY CUT: —
OTHER INFO:	PRINT PRESS POSITION IN TOP RIGHT CORNER OF GLUE FLAP IN 3 PT. HELVETICA BOLD		NO PRINT OR NO COATING AREA: — — COPY AREA: —



LICENSED

PERIOD 2014-2016 LIC. NO. 9591.73

16^{11/16}

Special Instructions for Cleaning and Decontamination Against HIV-1*, HBV* and HCV* of Surfaces or Objects Soiled With Blood or Body Fluids:
Personal Protection: When using this disinfecting towelette, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.
Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting towelette. A **MaxiWipe LA** towelette can be used for this purpose.
Contact Time: Thoroughly wet surface with a new **MaxiWipe LA** towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.
Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids must be autoclaved and disposed of in accordance with federal, State and local regulations for infectious materials disposal.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to pre-clean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals

CAUTION
 Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum, using tobacco, or using the toilet. Remove contaminated clothing and wash clothing before reuse.

FIRST AID
If in eyes: Hold eye open and rinse slowly and gently with water for 15-20 minutes. Remove contact lenses, if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for further advice.
If on skin: Take off contaminated clothing. Rinse skin immediately with plenty of water for 15-20 minutes. Call a poison control center or doctor for treatment advice.

For Chemical Emergency ONLY (spill, leak, fire, exposure or accident), call CHEMTREC at 1-800-424-9300. For ALL other inquiries about this product, call Henry Schein at 1-800-472-4346, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

STORAGE AND DISPOSAL
Towelette - Do not reuse towelette. Dispose of used towelette in trash.
Dispenser - Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available.

Distributed by:
 Henry Schein, Inc.
 135 Duryea Rd.
 Melville, NY 11747

MADE IN USA

HENRY SCHEIN®
MAXIWipe® LA
 Tuberculocidal • Bactericidal • Virucidal* • Fungicidal
DISINFECTING TOWELETTES

112-6843

MaxiWipe LA Disinfecting Towelettes are non-woven disposable towelettes intended for use in healthcare settings such as hospitals, dental offices, laboratories, clinics, ophthalmic offices, veterinary facilities and in kitchens** and bathrooms. MaxiWipe LA is effective against the following microorganisms on hard, non-porous surfaces when used as directed:

- *Mycobacterium tuberculosis* var: *bovis* (BCG) ■ *Staphylococcus aureus* ■ *Pseudomonas aeruginosa*
- *Salmonella enterica* ■ *Trichophyton mentagrophytes* ■ Methicillin Resistant *Staphylococcus aureus* (MRSA)
- Vancomycin Resistant *Enterococcus faecalis* (VRE) ■ *Staphylococcus aureus* with Reduced Susceptibility to Vancomycin ■ Hepatitis B Virus (HBV)* ■ Hepatitis C Virus (HCV)* ■ Herpes Simplex Virus Types 1 and 2*
- Human Immunodeficiency Virus (HIV-1)* ■ Influenza A2 Virus*

ACTIVE INGREDIENTS:

Diisobutylphenoxyethoxyethyl dimethyl benzyl ammonium chloride	0.28%
Isopropanol	17.20%
INERT INGREDIENTS:	82.52%
TOTAL	100.00% ¹

¹Does not include the weight of the dry towelette.



KEEP OUT OF REACH OF CHILDREN
CAUTION

Net Contents:
160 Pre-Saturated Towelettes (6" x 6.75")
 Net Weight: 1 lb. 13 oz.

See other panel for additional precautionary statements.
 EPA Reg. No. 46781-8-10597
 EPA Est. No. 56952-WI-001

DO NOT USE AS A BABY WIPE. NOT FOR USE ON SKIN.
DIRECTIONS FOR USE
 It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
MaxiWipe LA Disinfecting Towelettes are non-woven disposable towelettes intended for use on hard, non-porous surfaces in healthcare settings such as hospitals, emergency medical settings, neonatal units, surgical centers, isolation areas, out-patient surgical centers, nursing homes, dental offices, ophthalmic offices, eye surgical centers, patient care areas, dialysis centers, donation centers (blood, plasma, semen, milk, aphaeresis), veterinary facilities, salon settings, emergency vehicles, correctional facilities, schools, health club facilities, tanning salons, kitchens**, non-food contact surfaces in food preparation areas**, bathrooms, day care centers, police and fire vehicles, cadaver processing areas, laundry rooms, and other critical care areas where control of cross contamination between treated surfaces is required. **MaxiWipe LA** will clean and disinfect, when used as directed, such items as interior and exterior surfaces of infant incubators, bassinets, cribs and warmers, infant/child-care equipment surfaces, diaper changing stations, oxygen hoods, operating room tables and lights, laboratory equipment and surfaces, physical therapy equipment surfaces, ambulance equipment surfaces, slit lamps, hard, non-porous exterior surfaces of ultrasound transducers or probes, exam tables, gurneys, IV poles, dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, exterior surfaces of anesthesia machines and respiratory therapy equipment, SilkPeel™ with Dermalinfusion™, workstations, dental counter tops, loupes, exterior surfaces of endodontic equipment, such as apex locators, pulp testers and motors, stools, nail/ear care implements, tanning beds, salon surfaces, non-food contact surfaces**, exterior surfaces of refrigerator units**, exterior surfaces of appliances such as refrigerators, microwaves, food processors and stoves**, vanity tops**, floors, walls, handrails, door knobs, bed railings, tables, chairs, bathroom fixtures, exterior toilet surfaces, garbage cans, keyboards, carts, shopping carts, weight-lifting surfaces, drained whirlpool tank surfaces, cabinets, lights, telephones, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinyl chloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

Kitchen and Bath Wipes for Hard, Non-Porous Surfaces Cleans, Deodorizes and Kills Common Household Germs**

**A thorough rinse with potable water is required for surfaces in direct contact with food.
Dispenser Directions
 To start feed, remove cover and discard seal. From center of towelette roll, pull up a towelette corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first towelette and tear off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.
Cleaning Instructions:
 Use one **MaxiWipe LA** towelette to completely pre-clean surface of all gross debris.
For use as a virucide*, including HIV-1*, HBV* and HCV* applications and effectiveness Against Methicillin Resistant *Staphylococcus aureus* (MRSA), Vancomycin Resistant *Enterococcus faecalis* (VRE) and *Staphylococcus aureus* with Reduced Susceptibility to Vancomycin:
 Use a second **MaxiWipe LA** towelette to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes at room temperature (68°F/20°C).
For use as a disinfectant:
MaxiWipe LA is a two-step disinfectant product. One wipe is required to pre-clean then a second wipe is required to disinfect. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at room temperature (68°F/20°C) for Tuberculocidal activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella enterica*.
To disinfect external surfaces of ultrasound transducers:
 Pre-clean ultrasound gel from external transducer to be disinfected. Then thoroughly wet the probe with a **MaxiWipe LA** towelette. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at 68°F/20°C.
MaxiWipe LA effectively kills Human Immunodeficiency Virus Type 1 (HIV-1)*, Hepatitis B Virus (HBV)* and Hepatitis C Virus (HCV)* on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of HIV-1*, HBV* and HCV*.

4^{1/4}

← LOT CODE AREA: NO COPY →