

DO NOT REMOVE THIS INFORMATION!
IT IS IMPORTANT FOR PRODUCTION WORKFLOW. ROCKLINE

DIE LINE #	2732G	LABEL, 16 11/16" x 4 1/4" FOR 120 MM CANISTER	STACK (mm) N/A
SIZE	16-11/16 x 4-1/4 (+/- 1/32)		VIEW: PRINT SIDE
PRODUCT	120 MM CANISTER PRODUCT		KEY CUT: _____ NO PRINT OR NO COATING COPY AREA: _____ AREA: _____
OTHER INFO:	PRINT PRESS POSITION IN TOP RIGHT CORNER OF GLUE FLAP IN 3 PT. HELVETICA BOLD		



LICENSED
PERIOD 2015-2017 LIC. NO. **9591.75**

16^{11/16}

Special Instructions for Cleaning and Decontamination Against HIV-1* and HBV* of Surfaces or Objects Soiled With Blood or Body Fluids:
Personal Protection: When using this disinfecting wipe, wear appropriate barrier protection such as disposable protective gloves, protective gown, face mask and eye covering.
Cleaning Procedure: Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfecting wipe. A Darby Surface Disinfectant Wipes can be used for this purpose.
Contact Time: Thoroughly wet surface with a new Darby Surface Disinfectant Wipes. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes. This contact time will not be sufficient for some of the organisms listed on the label. Refer to label for contact times against other organisms.
Disposal Of Infectious Materials: Cleaning materials used that may contain blood or other body fluids must be autoclaved and disposed of in accordance with federal, state and local regulations for infectious materials disposal.

This product is not to be used as a terminal steilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization/high-level disinfection.

PRECAUTIONARY STATEMENTS
Hazards to Humans and Domestic Animals
CAUTION
Harmful if absorbed through the skin. Causes moderate eye irritation. Avoid contact with eyes, skin, or clothing. Wash hands before eating, drinking, chewing gum,



Darby Surface Disinfectant Wipes

Bactericidal | Virucidal* | Fungicidal | Tuberculocidal

Darby Surface Disinfectant Wipes are non-woven disposable wipes intended for use on hard, non-porous surfaces in healthcare settings such as dental offices, hospitals, laboratories and clinics. Darby Surface Disinfectant Wipes are effective against the following microorganisms when used as directed:

- *Mycobacterium tuberculosis* var: *bovis* (BCG)
- *Staphylococcus aureus*
- *Pseudomonas aeruginosa*
- *Salmonella enterica*
- *Trichophyton mentagrophytes*
- Methicillin Resistant *Staphylococcus aureus* (MRSA)
- Vancomycin Resistant *Enterococcus faecalis* (VRE)
- Hepatitis B Virus (HBV)*
- Herpes Simplex Virus Types 1 and 2*
- Human Immunodeficiency Virus (HIV-1)*

ACTIVE INGREDIENTS:
Disobutylphenoxethoxyethyl dimethyl benzyl ammonium chloride 0.28%
Isopropanol 17.20%
INERT INGREDIENTS†: 82.52%
TOTAL 100.00%
† Does not include the weight of the dry wipe.

**Net Contents: 1 lb. 13 oz.
160 Pre-Saturated Wipes (6" x 6.75")**

FPA Reg. No. 46781-R-90812 • FPA Est. No. 56952-WL001

9430502

KEEP OUT OF REACH OF CHILDREN
CAUTION
See other panel for additional precautionary statements.

9430502

DO NOT USE AS A BABY WIPE. Not for use on skin.

DIRECTIONS FOR USE
It is a violation of Federal law to use this product in a manner inconsistent with its labeling.
Darby Surface Disinfectant Wipes are intended for use on hard, non-porous surfaces in healthcare settings such as dental offices, hospitals, emergency medical settings, surgical centers, out-patient surgical centers (OPSC), patient care areas, and other critical care areas where control of cross contamination between treated surfaces is required.

Darby Surface Disinfectant Wipes will clean and disinfect, when used as directed, such items as dental unit instrument trays, operatory light switches, exterior surfaces of amalgamators and dental curing lights, operating room tables and lights, laboratory equipment and surfaces, exterior surfaces of anesthesia machines and respiratory therapy equipment, workstations, dental countertops, dental chairs, loupes, exterior surfaces of endodontic equipment, such as apex locators, pulp testers and motors, stools, floors, walls, handrails, door knobs, tables, chairs, faucets, bathroom fixtures, exterior toilet surfaces, garbage cans, computers, keyboards, cabinets and telephones, which are made of plastic (i.e., polycarbonate, polypropylene, polyvinylchloride, polystyrene), and use sites listed on this label consisting of non-porous vinyl, stainless steel, painted surfaces, Plexiglas®, Formica® and glass.

Dispenser Directions:
To start feed, remove cover and discard seal. From center of wipe roll, pull up a wipe corner, twist it into a point and thread it through the hole in the container cover. Pull through about one inch. Replace cover. Pull out first wipe and tear off at a 90° angle. Remaining wipes feed automatically, ready for next use. When not in use, keep center cap closed to prevent solution loss.

Cleaning Instructions:
Use one Darby Surface Disinfectant Wipes to completely pre-clean surface of all gross debris.
For use as a virucide*, including HIV-1* and HBV* applications and effectiveness against Methicillin Resistant *Staphylococcus aureus* (MRSA) and Vancomycin Resistant *Enterococcus faecalis* (VRE):
Use a second Darby Surface Disinfectant Wipes to thoroughly wet the surface. Repeated use of the product may be required to ensure that the surface remains visibly wet for 2 minutes at room temperature (68°F/20°C).

For use as a Disinfectant:
Darby Surface Disinfectant Wipes is a two-step disinfectant product. One wipe is required to pre-clean then a second wipe is required to disinfect. Repeated use of the product may be required to ensure that the surface remains visibly wet for 3 minutes at room temperature (68°F/20°C) for Tuberculocidal activity and effectiveness against *Staphylococcus aureus*, *Pseudomonas aeruginosa*, *Trichophyton mentagrophytes* and *Salmonella enterica*.

Darby Surface Disinfectant Wipes effectively kills Human Immunodeficiency Virus Type 1 (HIV-1)* and Hepatitis B Virus (HBV)* on pre-cleaned environmental surfaces/objects previously soiled with blood/body fluids in healthcare settings or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood/body fluids, and in which the surfaces/objects can be associated with the potential for transmission of HIV-1*, and HBV*.

STORAGE AND DISPOSAL
Wipe - Do not reuse wipe. Dispose of used wipe in trash.
Dispenser - Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available.

Distributed by:
Darby Dental Supply, LLC
300 Jericho Quadrangle
Jericho, NY 11753

To reorder:
800-645-2310 or Darby.com

70-3520-2
86411-22A



LOT CODE AREA: NO COPY

4^{1/4}

