

DO NOT REMOVE THIS INFORMATION!  
IT IS IMPORTANT FOR PRODUCTION WORKFLOW. ROCKLINE

DIE LINE #	2732G	LABEL, 16 11/16" x 4 1/4" FOR 120 MM CANISTER	STACK (mm) N/A
SIZE	16-11/16 x 4-1/4 (+/- 1/32)		VIEW: PRINT SIDE
PRODUCT	120 MM CANISTER PRODUCT		KEY CUT: — NO PRINT OR COPY AREA: NO COATING AREA: — —
OTHER INFO:	PRINT PRESS POSITION IN TOP RIGHT CORNER OF GLUE FLAP IN 3 PT. HELVETICA BOLD		

CAVIWIPES AF XL 80 US  
File: 70-5086-1 CWF\_XL\_80\_CANISTER\_US\_P7

Color: 100% Black, PMS 220, UV Varnish  
**OPTION 1 OF 2**

Grain Direction

FPO  
Name not confirmed

GLUE FLAP AREA:  
NO COATING FROM  
EDGE OF LABEL TO  
LINE

4 1/4

3 9/16

**KILLS HIV-1, HBV and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/ OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in healthcare settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV and HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.**

**Personal Protection:** When handling items soiled with blood or body fluids use disposable latex gloves, gowns, masks, and eye coverings.

**Cleaning Procedures:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product. A CaviWipes™ AF towelette can be used for this purpose.

**Contact Time:** Thoroughly wet surface with a new CaviWipes AF towelette. Allow surface to remain wet for 5 minutes and air dry.

**Disposal Of Infectious Materials:** Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.  
Not for personal cleansing. This is **not** a baby wipe! Do not flush in toilet.

**PRECAUTIONARY STATEMENTS**  
**Hazards to Humans and Domestic Animals**  
**CAUTION**  
Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

**FIRST AID**  
Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

**If in eyes:** Hold eyes open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

**STORAGE AND DISPOSAL**  
DO NOT CONTAMINATE WATER, FOOD, OR FEED BY STORAGE OR DISPOSAL.  
**PESTICIDE STORAGE/ DISPOSAL** - Store in an area inaccessible to children.  
**CONTAINER STORAGE/ DISPOSAL** - Wipes: Dispose of towelette in trash after use. Do not flush. Canister: Tightly close lid between uses to retain moisture. Nonrefillable container. Do not reuse or refill this canister. Offer empty canister for recycling if available or discard in trash.

CaviWipes AF has demonstrated effectiveness against Influenza A virus and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 Influenza A virus.

**REORDER No. 13-8100**

Manufactured For:  
METREX™ RESEARCH  
28210 Wick Road  
Romulus, MI 48174  
1-800-841-1428

For Chemical Emergency ONLY (spill, leak, fire, exposure, or accident), call CHEMTREC at 1-800-424-9300. For ALL other inquiries about this product, call Metrex Customer Service at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

For product or technical information, contact Metrex at 800-841-1428 or visit our website at [www.metrex.com](http://www.metrex.com).

**MADE IN USA**

70-5086-1 RL 81773-22

# CaviWipes AF™

## disinfecting towelettes

**Net Contents: 1 lbs. 10 oz. • 80 Pre-Moistened Towelettes (6" x 12")**

**• TUBERCULOCIDAL • PSEUDOMONICIDAL**  
**• BACTERICIDAL • \*VIRUCIDAL**

**CaviWipes AF** are non-woven disposable towelettes pre-saturated with a quaternary disinfectant cleaner. **CaviWipes AF** are intended for use in health care settings such as hospitals, dental offices, laboratories, clinics, ophthalmic offices, veterinary facilities and bathrooms.

**ACTIVE INGREDIENTS:**  
Alkyl (60% C<sub>14</sub>, 30% C<sub>16</sub>, 5% C<sub>12</sub>, 5% C<sub>18</sub>) dimethyl benzyl ammonium chlorides ..... 0.105%  
Alkyl (68% C<sub>12</sub>, 32% C<sub>14</sub>) dimethyl ethylbenzyl ammonium chlorides ..... 0.105%  
**INERT INGREDIENTS:** ..... 99.790%  
**TOTAL** ..... 100.000%

**KEEP OUT OF REACH OF CHILDREN**  
**CAUTION**  
See side panels for additional precautionary statements. EPA Reg. No. 1839-223-46781 EPA Est. No. 56952-WI-001

Department of Agriculture  
STATE OF HAWAII  
**LICENSED**  
PERIOD 2015-2017 LIC. NO.  
**9591.76**

LOT CODE AREA---

15

NO PRINT AREA

16+11/16

1 1/4

3/16

1/2

SPECS:  
Dimensions: 28.35" x 4.72"  
Color: 100% Black, PMS 220

Handle

Centre Frontside

Handle

Centre Backside

Handle



LICENSED  
PERIOD 2015-2017 LIC. NO.  
9591.76

# CaviWipes™ AF

## disinfecting towelettes

- TUBERCULOCIDAL
- BACTERICIDAL

- PSEUDOMONICIDAL
- \*VIRUCIDAL

CaviWipes™ AF are non-woven disposable towelettes pre-saturated with a quaternary disinfectant cleaner. CaviWipes™ AF are intended for use in health care settings such as hospitals, dental offices, laboratories, clinics, ophthalmic offices, veterinary facilities and bathrooms.

<b>ACTIVE INGREDIENTS:</b>	
Alkyl (60% C <sub>12</sub> , 30% C <sub>14</sub> , 5% C <sub>16</sub> , 5% C <sub>18</sub> ) dimethyl benzyl ammonium chlorides	0.105%
Alkyl (68% C <sub>12</sub> , 32% C <sub>14</sub> ) dimethyl ethylbenzyl ammonium chlorides	0.105%
<b>INERT INGREDIENTS:</b>	99.790%
<b>TOTAL</b>	<b>100.000%</b>

**Reorder Number**  
13-8075

KEEP OUT OF REACH OF CHILDREN

**CAUTION**

See side panels for additional precautionary statements.

EPA Reg. No. 1839-223-46781  
EPA Est. No. 56952-WI-001

**CaviWipes™ AF** is intended for use in: hospitals and healthcare settings: ambulances, anesthesia, CAT labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctor's office(s), Emergency Rooms, E.R., Home health care, Hospices, ICU, Laboratory, Laundry rooms, Neonatal Intensive Care Units (NICU), Newborn nursery, Nursing homes, Operating rooms, Pediatrics, Pediatric Intensive Care Units (PICU), Physical therapy, Physician's office(s), Ophthalmic offices, Radiology, Recovery Room, Rehabilitation, Respiratory therapy, Surgery, Transport vehicles, X-Ray, Critical Care Areas: CCU, Emergency Rooms, E.R., I.C.U. Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU), Surgery, Nursery, Daycare centers, veterinary clinics, animal hospitals, animal life science laboratories, kennels, pet animal kennels, breeding establishments, grooming establishments, pet animal quarters, zoos, pet shops, tack shops, animal holding areas, veterinary waiting rooms, veterinary examination rooms, veterinary X-ray rooms, veterinary operating rooms, animal transportation vehicles, beauty salons, barber shops, correctional facilities, funeral homes, mortuaries, schools, health clubs, gyms, athletic facilities, airports, transportation terminals.

CaviWipes™ AF will clean and disinfect hard, non-porous, surfaces of: bed railings, cabinets, carts, chairs, counters, exam tables, gurneys, isolettes, IV poles, stethoscopes, stretchers, tables, telephones, toilet seats, toilets, bathtubs, bathroom fixtures, shower stalls, sinks, tiles, vanity tops, hospital beds, traction devices, MRI, CAT, scales, paddles, wheelchairs, ultrasound transducers and probes, changing tables, cribs, diaper pails, cabinets, chairs, computers, countertops, desks, desktops, doorknobs, fax machines, garbage cans, headsets, keyboards, railings, seats, sinks, tables, tabletops, telephones, trash cans, gym equipment, exercise machines, wrestling mats, gymnastic equipment.

Use CaviWipes™ AF to clean and disinfect non-porous personal protective safety equipment, protective headgear, hard hats, half mask respirators, full face breathing apparatus, gas masks, goggles, spectacles, face shields, silicone rubber or PVC hearing protectors and vinyl covered ear muffs. Rinse all equipment that comes in prolonged contact with skin before reuse with clean warm water (about

120°F), and allow to air dry. (Caution: Cleaning at the 120°F temperature will avoid overheating and distortion of the personal safety equipment that would necessitate replacement.) CaviWipes™ AF is formulated to disinfect hard, non-porous, inanimate environmental surfaces such as floors, walls, metal surfaces, stainless steel surfaces, glazed porcelain, glazed ceramic tile, non-polycarbonate plastic surfaces, bathrooms, shower stalls, bathtubs, and cabinets.

**DIRECTIONS FOR USE**  
It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. **For plastic and painted surfaces, spot test on an inconspicuous area before use.**

**CLEANING/DEODORIZING:** Use one CaviWipes™ AF towelette to deodorize and completely remove all gross debris from surfaces to be treated. **For heavily soiled surfaces, clean before following disinfecting instructions.**

**DISINFECTING:** To disinfect hard, non-porous surfaces, thoroughly wet the surface to be treated. Treated surface must remain visibly wet for 5 minutes for disinfection:

- To disinfect against *Escherichia coli* (*E. coli*), *Escherichia coli* O157:H7, *Pseudomonas aeruginosa*, *Salmonella* (*choleraesuis*) *enterica* and Vancomycin Resistant *Enterococcus faecalis* (VRE), allow the treated surface to remain wet for 5 minutes.
- To disinfect against *Mycobacterium bovis* BCG (Tuberculosis) allow the treated surface to remain wet for 5 minutes.

- To disinfect against *Staphylococcus aureus* and Methicillin Resistant *Staphylococcus aureus* (MRSA), allow the treated surface to remain wet for 5 minutes.
- A potable water rinse is required for surfaces in direct contact with food.

**\*VIRUCIDAL ACTIVITY:** CaviWipes™ AF is an effective virucide on hard, non-porous surfaces against:

- Influenza A (H1N1) Virus (ATCC VR-1469, Strain A/PR/8/34), Hepatitis C Virus (HCV), Hepatitis B Virus (HBV), Bovine Viral Diarrhea Virus (BVDV) and Duck Hepatitis B Virus (DHBV) when the treated surface is allowed to remain wet for 5 minutes.
- Human Immunodeficiency Virus Type 1 (HIV-1) when the treated surface is allowed to remain wet for 2 minutes.

Use additional towelettes if needed to assure that the treated surface remains continuously wet for the prescribed time.

**For Ultrasound Transducers and Probes:**  
Use only on external surfaces of ultrasound transducers and probes. With a clean washcloth, clean gel from external surface of the ultrasound transducer or probe to be disinfected. Then thoroughly wet probe with towelettes. Treated surface must remain wet for 5 minutes. Use additional towelettes if needed to assure continuous 5 minute wet contact time. Let air dry.

**Veterinary Practice/ Animal Care Animal Laboratory/ Zoos/ Pet Shop/Kennels/ Grooming Establishments Disinfection Directions:** For cleaning and disinfecting the following hard non-porous surfaces: equipment not used for animal food or water, utensils, instruments, cages, kennels, countertops, tables, etc. Remove all animals and feed from premises, cages, crates, etc. Remove all litter, droppings and manure from equipment to be disinfected. Thoroughly clean all surfaces with soap or detergent and rinse with water. Thoroughly wet all surfaces to be treated with towelettes. Treated surfaces must remain wet for 5 minutes. Use additional towelettes if

needed to assure continuous wet contact time of 5 minutes. Let air dry. Ventilate buildings and other closed spaces. Do not house animals or employ equipment until treatment has been absorbed, set or dried.

**TO OPEN PACKAGE:** Flip open dispensing flap and remove lid by lifting upward. Place pouch in container. Open pouch by peeling back seal. Insert towelette through opening in lid. Replace lid. The next towelette pops up automatically. When finished, close lid to retain moisture.

**KILLS HIV-1, HBV and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS** in healthcare settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

**SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV and HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.**  
**Personal Protection:** When handling items soiled with blood or body fluids use disposable latex gloves, gowns, masks, and eye coverings.  
**Cleaning Procedures:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product. A CaviWipes™ AF towelette can be used for this purpose.  
**Contact Time:** Thoroughly wet surface with a new CaviWipes™ AF towelette. Allow surface to remain wet for 5 minutes and air dry.  
**Disposal Of Infectious Materials:** Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal.  
Not for personal cleansing. This is not a baby wipe! Do not flush in toilet.

**PRECAUTIONARY STATEMENTS**  
**Hazards to Humans and Domestic Animals**  
**CAUTION**  
Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.  
**FIRST AID**  
Have the product container or label with you when calling a poison control center or doctor, or going for treatment.  
**If in eyes:** Hold eyes open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

**REORDER No. 13-8075**  
Manufactured For:  
METREX™ RESEARCH  
28210 Wick Road  
Romulus, MI 48174  
1-800-841-1428

For Chemical Emergency ONLY (spill, leak, fire, exposure, or accident), call CHEMTREC at 1-800-424-9300. For All other inquiries about this product, call Metrex Customer Service at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

For product or technical information, contact Metrex at 800-841-1428 or visit our website at [www.metrex.com](http://www.metrex.com).

**MADE IN USA**



\*497113115044\*

**FPO**  
BARCODE  
TO COME

70-5090-1 RL 81773-22

28.35" / 720 mm

4.72" / 120 mm

SPECS:  
Dimensions: 13" x 13" (TBD NOT CONFIRMED)  
Color: 100% Black, PMS 2935 C

# FRONT PANEL



# CaviWipes™ AF

disinfecting  
towelettes

- TUBERCULOCIDAL
- BACTERICIDAL
- PSEUDOMONICIDAL
- \*VIRUCIDAL

**CaviWipes™ AF** are non-woven disposable towelettes pre-saturated with a quaternary disinfectant cleaner. **CaviWipes AF** are intended for use in health care settings such as hospitals, dental offices, laboratories, clinics, ophthalmic offices, veterinary facilities and bathrooms.

**ACTIVE INGREDIENTS:**

Alkyl (60% C<sub>14</sub>, 30% C<sub>16</sub>, 5% C<sub>12</sub>, 5% C<sub>18</sub>)  
dimethyl benzyl ammonium chlorides ..... 0.105%  
Alkyl (68% C<sub>12</sub>, 32% C<sub>14</sub>)  
dimethyl ethylbenzyl ammonium chlorides ..... 0.105%

**INERT INGREDIENTS:** ..... 99.790%  
TOTAL ..... 100.000%

**Reorder Number 13-8070**

KEEP OUT OF REACH OF CHILDREN  
**CAUTION**

EPA Reg. No. 1839-223-46781  
EPA Est. No. 56952-WI-001



**Net Contents: 5 lbs. 14 oz.**  
**200 Pre-Moistened Towelettes (7" x 15")**



13 in

13 in

SPECS:  
Dimensions: 13" x 13" (TBD NOT CONFIRMED)  
Color: 100% Black, PMS 2935 C

# BACK PANEL

**CaviWipes™ AF** is intended for use in: hospitals and healthcare settings: ambulances, anesthesia, CAT labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctor's office(s), Emergency Rooms, E.R., Home health care, Hospices, ICU, Laboratory, Laundry rooms, Neonatal Intensive Care Units (NICU), Newborn nursery, Nursing homes, Operating rooms, Pediatrics, Pediatric Intensive Care Units (PICU), Physical therapy, Physician's office(s), Ophthalmic offices, Radiology, Recovery Room, Rehabilitation, Respiratory therapy, Surgery, Transport vehicles, X-Ray, Critical Care Areas: CCU, Emergency Rooms, E.R., I.C.U. Neonatal Intensive Care Units (NICU), Operating Rooms, Pediatric Intensive Care Units (PICU), Surgery, Nursery, Daycare centers, veterinary clinics, animal hospitals, animal life science laboratories, kennels, pet animal kennels, breeding establishments, grooming establishments, pet animal quarters, zoos, pet shops, tack shops, animal holding areas, veterinary waiting rooms, veterinary examination rooms, veterinary X-ray rooms, veterinary operating rooms, animal transportation vehicles, beauty salons, barber shops, correctional facilities, funeral homes, mortuaries, schools, health clubs, gyms, athletic facilities, airports, transportation terminals.

**CaviWipes AF** will clean and disinfect hard, non-porous, surfaces of: bed railings, cabinets, carts, chairs, counters, exam tables, gurneys, isolettes, IV poles, stethoscopes, stretchers, tables, telephones, toilet seats, toilets, bathtubs, bathroom fixtures, shower stalls, sinks, tiles, vanity tops, hospital beds, traction devices, MRI, CAT, scales, paddles, wheelchairs, ultrasound transducers and probes, changing tables, cribs, diaper pails, cabinets, chairs, computers, countertops, desks, desktops, doorknobs, fax machines, garbage cans, headsets, keyboards, railings, seats, sinks, tables, tabletops, telephones, trash cans, gym equipment, exercise machines, wrestling mats, gymnastic equipment.

Use **CaviWipes AF** to clean and disinfect non-porous personal protective safety equipment, protective headgear, hard hats, half mask respirators, full face breathing apparatus, gas masks, goggles, spectacles, face shields, silicone rubber or PVC hearing protectors and vinyl covered ear muffs. Rinse all equipment that comes in prolonged contact with skin before reuse with clean warm water (about 120°F), and allow to air dry. (Caution: Cleaning at the 120°F temperature will avoid overheating and distortion of the personal safety equipment that would necessitate replacement.)

**CaviWipes AF** is formulated to disinfect hard, non-porous, inanimate environmental surfaces such as floors, walls, metal surfaces, stainless steel surfaces, glazed porcelain, glazed ceramic tile, non-polycarbonate plastic surfaces, bathrooms, shower stalls, bathtubs, and cabinets.

#### DIRECTIONS FOR USE

It is a violation of Federal Law to use this product in a manner inconsistent with its labeling.

This product is not to be used as a terminal sterilant/high-level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contacts intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body.

**For plastic and painted surfaces, spot test on an inconspicuous area before use.**

**CLEANING/DEODORIZING:** Use one **CaviWipes AF** towelette to deodorize and completely remove all gross debris from surfaces to be treated.

**For heavily soiled surfaces, clean before following disinfecting instructions.**

**DISINFECTING:** To disinfect hard, non-porous surfaces, thoroughly wet the surface to be treated. Treated surface must remain visibly wet for 5 minutes for disinfection:

- To disinfect against *Escherichia coli* (*E. coli*), *Escherichia coli* O157:H7,

*Pseudomonas aeruginosa*, *Salmonella (choleraesuis) enterica* and Vancomycin Resistant *Enterococcus faecalis* (VRE), allow the treated surface to remain wet for 5 minutes.

- To disinfect against *Mycobacterium bovis* BCG (Tuberculosis) allow the treated surface to remain wet for 5 minutes.
- To disinfect against *Staphylococcus aureus* and Methicillin Resistant *Staphylococcus aureus* (MRSA), allow the treated surface to remain wet for 5 minutes.

A potable water rinse is required for surfaces in direct contact with food.

**\*VIRUCIDAL ACTIVITY:** **CaviWipes AF** is an effective virucide on hard, non-porous surfaces against:

- Influenza A (H1N1) Virus (ATCC VR-1469, Strain A/PR/8/34), Hepatitis C Virus (HCV), Hepatitis B Virus (HBV), Bovine Viral Diarrhea Virus (BVDV) and Duck Hepatitis B Virus (DHBV) when the treated surface is allowed to remain wet for 5 minutes.
- Human Immunodeficiency Virus Type 1 (HIV-1) when the treated surface is allowed to remain wet for 2 minutes.

Use additional towelettes if needed to assure that the treated surface remains continuously wet for the prescribed time.

#### For Ultrasound Transducers and Probes:

Use only on external surfaces of ultrasound transducers and probes.

With a clean washcloth, clean gel from external surface of the ultrasound transducer or probe to be disinfected. Then thoroughly wet probe with towelettes. Treated surface must remain wet for 5 minutes. Use additional towelettes if needed to assure continuous 5 minute wet contact time. Let air dry.

#### Veterinary Practice/ Animal Care Animal Laboratory/ Zoos/ Pet Shop/Kennels/ Grooming Establishments Disinfection Directions:

For cleaning and disinfecting the following hard non-porous surfaces: equipment not used for animal food or water, utensils, instruments, cages, kennels, countertops, tables, etc. Remove all animals and feed from premises, cages, crates, etc. Remove all litter, droppings and manure from equipment to be disinfected. Thoroughly clean all surfaces with soap or detergent and rinse with water. Thoroughly wet all surfaces to be treated with towelettes. Treated surfaces must remain wet for 5 minutes. Use additional towelettes if needed to assure continuous wet contact time of 5 minutes. Let air dry. Ventilate buildings and other closed spaces. Do not house animals or employ equipment until treatment has been absorbed, set or dried.

#### TO OPEN:

**TO OPEN PACKAGE:** Flip open dispensing flap and remove lid by lifting upward. Place refill in container. Open in container by peeling back seal. Insert towelette through opening in lid. Replace lid. The next towelette pops up automatically. When finished close lid to retain moisture.

#### KILLS HIV-1, HBV and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS

in healthcare settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV) and Hepatitis C Virus (HCV).

#### SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1, HBV and HCV ON SURFACES/OBJECTS SOILED WITH BLOOD/BODY FLUIDS.

**Personal Protection:** When handling items soiled with blood or body fluids use disposable latex gloves, gowns, masks, and eye coverings.

**Cleaning Procedures:** Blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this product. A **CaviWipes™ AF** towelette can be used for this purpose.

**Contact Time:** Thoroughly wet surface with a new **CaviWipes AF** towelette. Allow surface to remain wet for 5 minutes and air dry.

**Disposal Of Infectious Materials:** Blood and other body fluids must be autoclaved and disposed of according to local regulations for infectious waste disposal. Not for personal cleansing. This is **not** a baby wipe! Do not flush in toilet.

#### PRECAUTIONARY STATEMENTS

##### Hazards to Humans and Domestic Animals

##### CAUTION

Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco or using the toilet.

##### FIRST AID

Have the product container or label with you when calling a poison control center or doctor, or going for treatment.

**If in eyes:** Hold eyes open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.

## STORAGE AND DISPOSAL

DO NOT CONTAMINATE WATER, FOOD, OR FEED BY STORAGE OR DISPOSAL.

**PESTICIDE STORAGE** - Store in an area inaccessible to children.

**CONTAINER STORAGE/ DISPOSAL** - Wipes: Dispose of towelette in trash after use. Do not flush. Container: Refill only with **CaviWipes AF**. Do not reuse or refill except as described in the directions for use. Tightly close lid between uses to retain moisture. Offer empty container for recycling if available or discard in trash. Refill package: Nonrefillable package. Do not reuse or refill this package. Offer empty package for recycling if available or discard in trash.

**CaviWipes AF** has demonstrated effectiveness against Influenza A virus and is expected to inactivate all Influenza A viruses including Pandemic 2009 H1N1 Influenza A virus.

#### REORDER No. 13-8070

Manufactured For:  
METREX™ RESEARCH  
28210 Wick Road.,  
Romulus, Michigan 48174  
1-800-841-1428

For Chemical Emergency ONLY (spill, leak, fire, exposure, or accident), call CHEMTREC at 1-800-424-9300. For ALL other inquiries about this product, call Metrex Customer Service at 1-800-841-1428, Monday through Friday, between 6 a.m. and 4 p.m. Pacific Time.

For product or technical information, contact Metrex at **800-841-1428** or visit our website at [www.metrex.com](http://www.metrex.com).

#### MADE IN USA

70-5092-1      RL 81773-22

13 in