

Die Line # 222 (Thick Canister Wrap around label)



STATE OF HAWAII
Department of Agriculture

ACCEPTED

LICENSE NO. **9960.3**

DisCide® **ULTRA**

DISINFECTING TOWELETTES

A One-Step Disinfectant - Cleaner - Decontaminant

Kills All Listed Pathogens IN ONE MINUTE

◆ Bactericidal ◆ Tuberculocidal ◆ Virucidal* ◆ Fungicidal ◆ Herpeticidal

DisCide ULTRA Disinfecting Towelettes are non-woven disposable towelettes impregnated with a dual chain quaternary ammonium/high level alcohol solution. *Healthcare Use* in Health Care & Alternate Care Settings such as hospitals, intensive care units, emergency medical settings, central supply, laboratories, clinics, nursing homes, and dental suites where control of the cross-contamination hazards between treated surfaces is required. *Consumer Use* on hard, non-food-contact surfaces in kitchens, food service areas and bathrooms and other related hard, non-food-contact surfaces such as shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, and door knobs and handles. DisCide ULTRA Disinfecting Towelettes eliminate odors and are effective against the following pathogens on hard non-porous, contact surfaces when used as directed:

- Tuberculosis bacteria (*Mycobacterium bovis*) (1 min. at 20°C)
- Staphylococcus aureus (Staph) (1 min.)
- Pseudomonas aeruginosa (1 min.)
- Salmonella enterica (1 min.)
- Escherichia coli (E. coli) (1 min.)
- Avian Influenza A Virus* (H3N2 strain) (30 sec.)
- Influenza A Virus* (Hong Kong strain) (30 sec.)
- Herpes Simplex Virus type 2* (genital herpes virus) (1 min.)
- Candida albicans (1 min.)
- Proteus mirabilis (1 min.)
- MRSA (Methicillin Resistant Staphylococcus aureus) (1 min.)
- VRE (Vancomycin Resistant Enterococcus faecalis) (1 min.)
- HBV* (Hepatitis B Virus) (1 min.)
- HCV* (Hepatitis C Virus) (1 min.)
- RSV* (Respiratory Syncytial Virus) (1 min.)
- HIV-1* (AIDS virus) (1 min.)
- Human Coronavirus* (30 sec.)
- Adenovirus type 2* (1 min.)
- Pandemic 2009 H1N1 Influenza A Virus* (30 sec.) (formerly called swine flu)

DisCide ULTRA Disinfecting Towelettes meet the disinfection requirements of OSHA's Bloodborne Pathogen Standard (29 CFR §1910.1030).



NET CONTENTS: 1 lb 5 oz
60 TOWELETTES (10.5 X 10 IN.)

ACTIVE INGREDIENTS:

n-alkyl (60% C14, 30% C16,
5% C12, 5% C18) dimethyl benzyl
ammonium chloride 0.12%

n-alkyl (68% C12, 32% C14)
dimethyl ethyl benzyl ammonium
chloride 0.12%

Isopropyl alcohol 63.25%

OTHER INGREDIENTS 36.51%

TOTAL INGREDIENTS 100.00%

Keep Out Of Reach
Of Children
WARNING

See back panel for additional
precautionary statements

EPA Reg. No. 10492-4 • EPA Est. No. 56952-WI-001

Reorder: 10DIS

4.25"

10 751

DIRECTIONS FOR USE

81062-381E

It is a violation of Federal law to use this product in a manner inconsistent with its labeling.

GENERAL PRECAUTIONS AND RESTRICTIONS

This product is not to be used as a terminal sterilant/high level disinfectant on any surface or instrument that (1) is introduced directly into the human body, either into or in contact with the bloodstream or normally sterile areas of the body, or (2) contact intact mucous membranes but which does not ordinarily penetrate the blood barrier or otherwise enter normally sterile areas of the body. This product may be used to preclean or decontaminate critical or semi-critical medical devices prior to sterilization or high level disinfection.

TO START FEED: Remove large cover from container and discard seal. From center of towelette roll, pull up a towelette corner, twist into a point and thread through hole in container cover. Pull through about one inch. Replace cover. Pull out first towelette and snap off at a 90° angle. Remaining towelettes feed automatically, ready for next use. When not in use, keep small center cap closed to prevent moisture loss.

TO CLEAN, DECONTAMINATE, AND DISINFECT: Remove gross dirt and soil. Thoroughly wet the surface with a DisCide ULTRA Disinfecting Towelette and allow to remain wet for the required contact time.

Healthcare FOR DISINFECTION AND DEODORIZATION OF: inanimate, hard, non-porous surfaces such as chairs, stools, counter tops, drawer pulls, door knobs and handles, carts, baskets, tables, cabinets, lighting fixtures, bassinets, bed rails, incubators, physical therapy equipment, shower stalls, toilet exteriors and toilet seats, telephones and other related hard surfaces in hospitals, operating and surgical rooms, intensive and critical care units, emergency units, dental and veterinary offices, surgery, recovery, anesthesia, X-Ray, Cat. Lab, blood collection and donation centers, isolation units, orthopedics, newborn nursery, respiratory therapy, radiology, pediatrics, physical therapy, rehabilitation, ophthalmology, optometry, hospices, central supply, emergency medical vehicles, police and fire vehicles, correctional facilities, funeral homes, morgues, spas, health clubs, salons, schools and school medical facilities, school buses, day care and non-food-contact surfaces. DisCide ULTRA Disinfecting Towelettes may be used to preclean or decontaminate medical instruments prior to sterilization or high-level disinfection.

Consumer FOR DISINFECTION AND DEODORIZATION OF: inanimate, hard, non-porous kitchen/food service and bathroom surfaces such as counter tops, drawer pulls, carts, baskets, tables, cabinets, door knobs and handles, appliance exteriors, shower stalls, toilet exteriors and toilet seats, bathroom and lighting fixtures, sinks, flooring, walls, vanities, diaper pails, trash cans, telephones, shopping carts, child-restraint seats, exercise and weight-lifting equipment, office equipment, computer screens and key pads, telephones, cell phones, and other related hard, non-food-contact surfaces.

Use on stainless steel, glass, formica and other similar surfaces. Use care when using on varnished surfaces or plexi-glass. **DO NOT USE AS A BABY WIPE.** Discard towelette after use.

*Virucidal Activity:

DisCide Ultra Disinfecting Towelettes are effective against the following viruses in 30 seconds or one minute when used according to label directions on hard, non-porous environmental surfaces: MRSA, VRE, Avian Influenza A (H3N2 strain), Influenza A (Hong Kong strain), Pandemic 2009 H1N1 influenza A virus (formerly called swine flu), RSV, HBV, HCV, Human coronavirus, HIV-1, Adenovirus type 2, and Herpes Simplex virus type 2.

Kills HIV-1, HBV, and HCV ON PRE-CLEANED ENVIRONMENTAL SURFACES/OBJECTS PREVIOUSLY SOILED WITH BLOOD/BODY FLUIDS in health care settings (Hospitals, Nursing Homes) or other settings in which there is an expected likelihood of soiling of inanimate surfaces/objects with blood or body fluids, and in which the surfaces/objects likely to be soiled with blood or body fluids can be associated with the potential for transmission of Human Immunodeficiency Virus Type 1 (HIV-1) (associated with AIDS), Hepatitis B Virus (HBV), and Hepatitis C Virus (HCV).

SPECIAL INSTRUCTIONS FOR CLEANING AND DECONTAMINATION AGAINST HIV-1 (AIDS VIRUS), HBV, and HCV FOR SURFACES OR OBJECTS SOILED WITH BLOOD OR BODY FLUIDS

PERSONAL PROTECTION: Wear disposable latex gloves, protective gown, face mask and eye covering, as appropriate, when handling HIV-1, HBV, and HCV infected blood or body fluids.

CLEANING PROCEDURE: All blood and other body fluids must be thoroughly cleaned from surfaces and objects before application of this disinfectant. A DisCide ULTRA Disinfecting Towelette can be used for this purpose.

CONTACT TIME: Thoroughly wet surface with a DisCide ULTRA Disinfecting Towelette. Allow surfaces to remain wet for 1 minute, then air dry. The efficacy of a 1 minute contact time has been shown to be adequate against HIV-1 (AIDS virus), HBV, and HCV.

DISPOSAL OF INFECTIOUS MATERIALS: Blood and other body fluids should be autoclaved and disposed of according to Federal, State, and local regulations for infectious waste disposal.

PRECAUTIONARY STATEMENTS:

Hazards to Humans and Domestic Animals: WARNING. Causes substantial but temporary eye injury. Do not get in eyes or on clothing. Wear protective eyewear. Wash thoroughly with soap and water after handling. Remove contaminated clothing and wash clothing before reuse.

Physical or Chemical Hazards: Flammable. Keep away from heat or open flame.

STORAGE AND DISPOSAL

Do not contaminate water, food, or feed by storage and disposal.

PESTICIDE STORAGE: Store away from heat.

CONTAINER DISPOSAL: Nonrefillable container. Do not reuse or refill this container. Offer for recycling, if available. Dispose of used towelette in trash.

FIRST AID

If in eyes:

- Hold eye open and rinse slowly and gently with water for 15-20 minutes.
- Remove contact lenses, if present, after the first 5 minutes, then continue rinsing eye.
- Call a poison control center or doctor for treatment advice.

HOT LINE NUMBER

Have the product container or label with you when calling a poison control center or doctor, or going for treatment. You may also contact Chem-Tel Inc. at 1-800-255-3924.

For technical information on DisCide ULTRA Spray, Towelettes, Aerosols, and DisCide XRA Hand Sanitizing Wipes, call 1-800-344-6424 or fax 203-377-8988.

Palmero Health Care 120 Goodwin Place, Stratford, CT 06615 www.palmerohealth.com